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Dear Customer,

*Thank you for purchasing a MAVIG Portegra2 suspension system or component.
Please read this document carefully and follow all directions, warnings, and notices.
Feel free to contact us with any questions or comments on our products.*

Contact Information

For questions or comments, please see back of manual for nearest MAVIG contact.

Disclaimers

Considerable effort has been made to ensure that the contents of this manual are free from omissions and inaccuracies. However, MAVIG makes no warranties with respect to the contents of this document and disclaims any implied warranties.

Changes

MAVIG products are under continuous development. MAVIG reserves the right to modify the manual, packaging, equipment or technical specifications, and scope of delivery without prior notice.

Translations

The master document is in English. In case of translation, precedence remains with the English version.

Copyright and Trademarks

All rights reserved. These instructions/manuals are protected by copyright law. MAVIG and Portegra2 are registered trademarks of the company MAVIG GmbH. All other trademarks mentioned in the manuals are the property of the corresponding manufacturer.

The logo for Portegra2, featuring the word "PORTEGRA" in a grey, sans-serif font, followed by a large, stylized orange "2".

© 10/2013 MAVIG GmbH

Introduction	02
Content	03
1.0 Symbols and Definitions	04-05
2.0 Delivery	05
2.1 Transport and Storage Conditions	05
2.2 Tools required for Assembly	05
3.0 Important Safety and Precautionary Comments	06-07
4.0 Mounting the track system to the ceiling	08-13
4.1 Mounting the ceiling track	08-09
4.2 Electrical connection of the carriage to the cable spooler	10-11
4.3 Electrical connection of the cable spooler	12-13
4.4 Mounting the carriage to the track system	12-13
5.0 Installing a Portegra2 column to a MAVIG mounting plate	14-17
5.1 Installation of the MAVIG mounting plate to Unistrut® rails	14-15
5.2 Installation of the MAVIG mounting plate to a ceiling	14-15
5.3 Installation of the ceiling column to the mounting plate	16-17
6.0 Installing a Portegra2 column to a ceiling	18-21
6.1 Installation of the safety chain with safety anchor	18-19
6.2 Installation of the ceiling column to a ceiling	20-21
7.0 Installation/removal of plastic covers (mounting posts)	22-23
7.1 Installation	22-23
7.2 Removal	22-23
8.0 Installation of the wall mount bracket	24-25
8.1 Installing the wall mount bracket	24-25
8.2 Electrical connection	24-25
9.0 Installation of the extension/spring arm combination	26-31
9.1 Installation of the extension/spring arm combination to the upper post	26-27
9.2 Installation of the extension/spring arm combination to the lower post of a 360° column - stationary version	28-29
9.3 Installation of the extension/spring arm combination to the lower post of a 360° column - carriage/track version	30-31
10.0 Installation and removal of adaptable elements	32-43
10.1 Installation of non-electrical or externally cabled end devices	32-35
10.2 Installation of a Surgical Lamp to the Extension/Spring Arm	36-39
10.3 Adjusting the weight capacity of the arm	40-41
10.4 Adjusting the vertical movement of the spring arm	40-41
10.5 Removal of a Device from the End of the Suspension Arm	42-43
11.0 Technical Data	44-47
11.1 Portegra2 Component Weights	44-45
11.2 Suspension Arm Load Capacity	44-45
11.3 Possible Spring Arm Load Classes	45
11.4 Additional Portegra2 Characteristic	47
11.5 Standards	47
11.6 Approved Products	47
12.0 CT Room Planning Considerations/Guide	48-49
12.1 Distance from the ceiling	48-49
12.2 Ceiling Column Selection	48-49
12.3 Ceiling Plate Placement (stationary version)	48-49
12.4 Track placement	48-49
Technical Drawings	50-51
Drilling Template	105

Usage

This manual is intended for the installation of the different components of the Portegra2 System produced by MAVIG. The instructions are for installing the Portegra2 ceiling track, mounting plate, wall mount, ceiling columns of the 360° and Twin model series in both fixed mounted and track/trolley mounted versions.

1.0 Symbols and Definitions

WARNING

Indicates that the information is a warning. Warnings advise you of circumstances that could result in injury or death to the patient or operator. Read and understand the warnings before installing or operating the system.

CAUTION

Indicates that the information is a caution. Cautions advise you of circumstances that could result in damage to the device or improper functioning of the device. Read and understand the cautions before installing or operating the system.

NOTE:

Provides usage tips and useful information.

WARNING

Indicates a possible electrical hazard, which can lead to severe injury or even death.

This symbol marks the points, where protective ground lines are attached.

This symbol represents to consult accompanying documents.

This symbol indicates possible collision points between two Portegra2 parts in this manual.

MAXIMUM WEIGHT LOAD SYMBOL

This symbol is combined with a weight in kilograms that shows the maximal weight that the marked component can hold.

**CE MARK**

With this symbol, MAVIG declares that the products comply with the relevant regulations set forth in the applicable European Directives.

**RECOGNISED UL COMPONENT**

This component has been recognised by Underwriters Laboratories Inc. Representative samples of this product have been reviewed by UL and comply with the applicable requirements.

**COLLISION - WARNING**

Collisions with other equipment can cause structural failure of the suspension arm, resulting in operator or patient injury. If collision occurs, immediately have the arm inspected by a qualified service technician.

2.0 Delivery

2.1 Transport and Storage Conditions

The following storage conditions apply for up to 15 weeks:

- Temperature: -25 to +70 °C
- Relative humidity 10 to 75%
- Air Pressure: 500 to 1060 hPa

Store only in closed or covered spaces. During use, the values of the operating conditions in the operating instructions apply.

2.2 Tools required for Assembly

- Loctite® 270 (included with Portegra2 suspension arms)
- Loctite® 243, or equivalent (not included)
- Flat head screwdriver with magnetic tip
- Philips screwdriver with magnetic tip
- Torque Wrench (62-195 in lbs or 7-22 Nm), or equivalent
- 3 mm, 4 mm, 5 mm, 6 mm, 8 mm Allen Wrench (metric)

3.0 Important Safety and Precautionary Comments

Only qualified, licensed technicians can perform electrical connections, installation, removal and repair. It is strongly recommended that at least two persons perform the installation.

Installing the system: Prior to installation, MAVIG emphasizes that a structural engineer must certify that the rough ceiling is strong enough to provide proper support for the entire system and any attached end devices. Installation must be completed according to local building codes. Failure to follow these procedures will release MAVIG from responsibility for damages of any sort.

Determination of required installation hardware and torque values for installation of the ceiling column and ceiling track is the sole responsibility of the structural engineer.

Ceiling mounted systems must be installed properly. Failure to follow the instructions provided may lead to a potentially dangerous and unstable condition of the system.

MAVIG is not responsible for unauthorized modifications made to the system or use of the system for unintended purposes. MAVIG cannot be held liable for improper operation and modifications. Since improper modifications may impair proper operation, safety or reliability of the system, product modifications require written authorization from MAVIG.

This product must be serviced with MAVIG approved, original replacement parts. Failure to follow this recommendation will void the manufacturer's warranty and will release MAVIG from all liability and warranty claims.

WARNING - Electric Shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

WARNING - Improper Installation.

Only qualified, licensed technicians can perform electrical connections, installation, removal, or repair of Portegra2 parts that have electrical components.

WARNING - Spring Under Constant Tension.

The Portegra2 spring arm is under constant spring tension. When installing and removing devices to and from the spring arm, always raise it up slowly to the upper stop. Failure to follow this instruction will void the manufacturer's warranty and release the manufacturer from all liability and warranty claims.

The system must only be operated by trained personnel. It is essential to have personnel understand that collisions of the system with other equipment can lead to severe damages, and failure of system.

The Portegra2 system is only validated and approved by MAVIG for the following CE-devices: radiation protective shields, monitor display holders, injector holders, and Dr. Mach OR and examination lights.

Portegra2 must not be combined with suspension systems from other manufacturers.

**WARNING - Improper Installation.**

All components must be installed or removed as described in this manual. It is prohibited to loosen any screws that are not mentioned in this manual, as they are secured with an appropriate thread locking adhesive and fixed with the appropriate torque. Unauthorized loosening of screws can weaken the system and result in damage or injury.

**WARNING - Do not install in MRI rooms!**

Portegra2 systems are made of ferrous materials. In a strong magnetic field, the system will be forcefully pulled in an unpredictable manner.

The Portegra2 System can be installed to a MAVIG ceiling mounting plate, MAVIG track system, Unistrut® rails, or directly to ceiling. Prior to installation, a structural engineer must certify that the rough ceiling is sufficiently strong to provide proper support to the system, and the structural engineer must select the appropriate installation hardware and its torque values.

It is strongly recommended that at least two people perform the installation!

Installation of the track system, mounting plate and ceiling column, as well as the appropriate installation hardware, must be completed according to local building codes. All local work and safety regulations must be observed. Verify that installation complies with local requirements.

**WARNING - Uneven Mounting Surface.**

A mounting surface that is not level, plumb or structurally sufficient can cause the overhead arm to drift out of position or system to fall, resulting in operator or patient injury. Ensure mounting surface is installed level or plumb.

**CAUTION - Incorrect Positioning.**

To avoid collisions the determination of the column or track position in the room (relative to the patient examination table) is based on following considerations: Fig. 1.1 and Fig. 1.2.



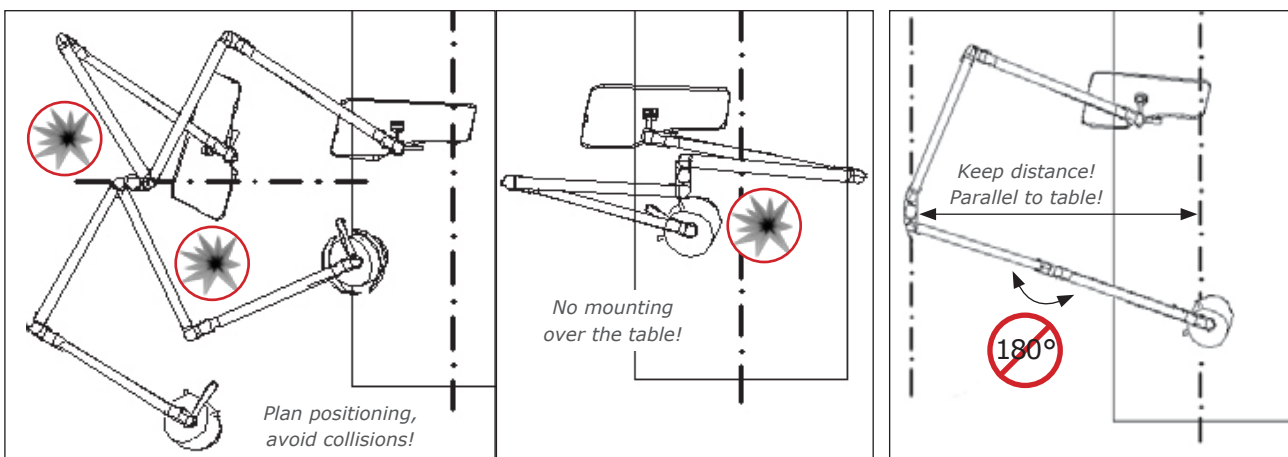
Fig 1.1 Please, carefully plan the position of the mounting assembly. Minimize the risk of the extension/spring arm colliding with the column, and other equipment.

Fig 1.2 Ensure no force can be exerted in the axial direction within the operation area of the extension/spring arm (at a 180° positioning).

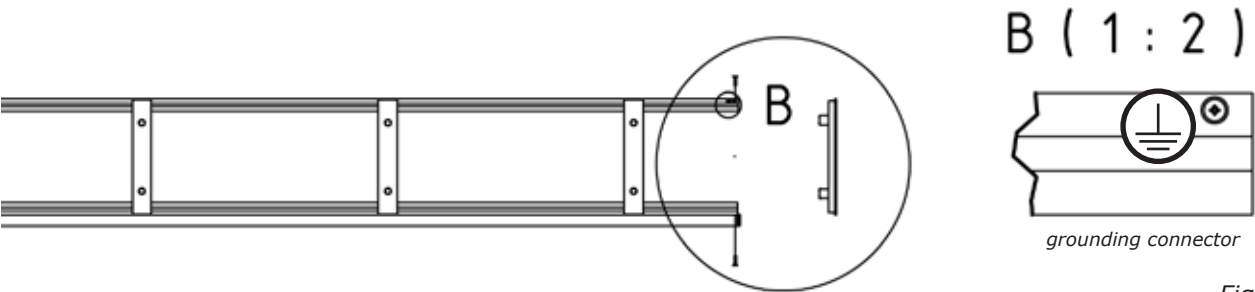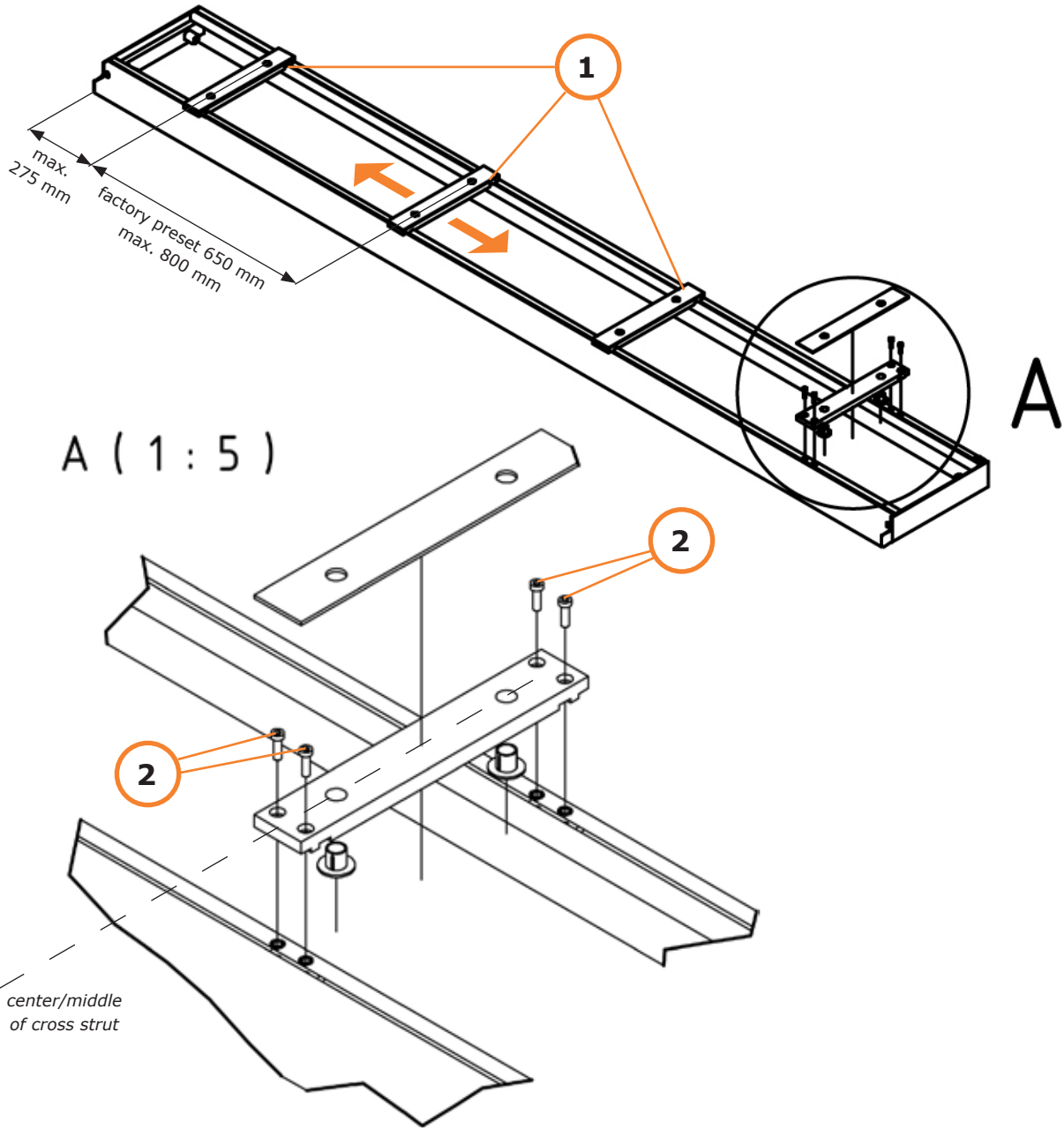4.0 Mounting the track system to the ceiling

4.1 Mounting the ceiling track

Selection of appropriate installation hardware and its torque value is the sole responsibility of the structural engineer. They will also define the system's layout.

NOTE: It is strongly recommended that at least two people perform the installation.

CAUTION - Ceiling Track Gauge.

The gauge of the ceiling track is preset at the factory. Do not loosen all cross struts simultaneously to avoid altering the gauge (Fig. 2).

- 4.1.1 It may be necessary to adjust the distance between cross struts (1/Fig. 2) to line up with the mounting points determined during the preplanning process.
- 4.1.2 To do this, loosen the four screws (2/Fig. 2) that hold each cross strut in place.
- 4.1.3 Now, the cross strut (1/Fig. 2) can be moved along the track and can be adjusted to the required intervals.
- 4.1.4 The maximum distance from the last attachment point, measured from the drill hole center (middle of the cross strut; 1/Fig. 2), to the ceiling track end must not exceed 275 mm (Fig. 2).
- 4.1.5 The distance between the cross struts, measured from the center of the cross strut, is preset at the factory at 650 mm. This distance must not exceed 800 mm (Fig. 2)
- 4.1.6 Apply Loctite® 221 to all cylinder head screws (2/Fig. 2) and tighten with 10 Nm torque to keep the cross struts in place.

CAUTION - Improper Cross Strut Placement.

By factory, these screws are secured with Loctite 221®. If repositioning of cross struts is needed, always reapply Loctite 221® on the cleaned screws.

- 4.1.7 If you have a Unistrut® system in place, attach the appropriate installation hardware to the Unistrut® rails, as determined during the planning process. Otherwise, hold the track up and mark the drill holes at the selected position, and drill as determined by the structural engineer.
- 4.1.8 If required, use the designated grounding point to ground the ceiling track (Fig. 3 B).
- 4.1.9 Lift the ceiling track and mount the track system to the Unistrut® rail at the previously adjusted cross struts or the previously drilled installation holes.
- 4.1.10 Mount the track to the ceiling with the selected mounting hardware. Secure the mounting screws finger-tight. Ensure the perfect horizontal positioning of the ceiling track using a block level. Use shims if necessary.
- 4.1.11 After leveling the ceiling track, tighten the screws to the torque value specified by the structural engineer and check again the horizontal positioning using a block level.

If you have an electrical installation with a cable spooler, continue with step 4.2. Otherwise proceed to step 4.3 - Mounting the carriage to the track system.

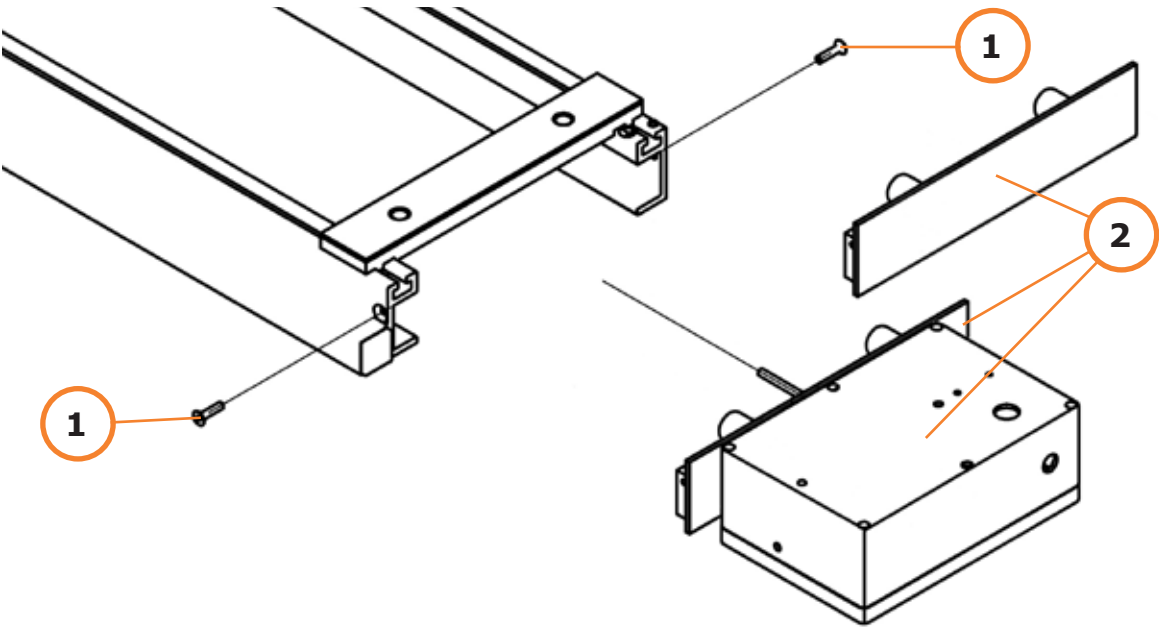
Fig. 4

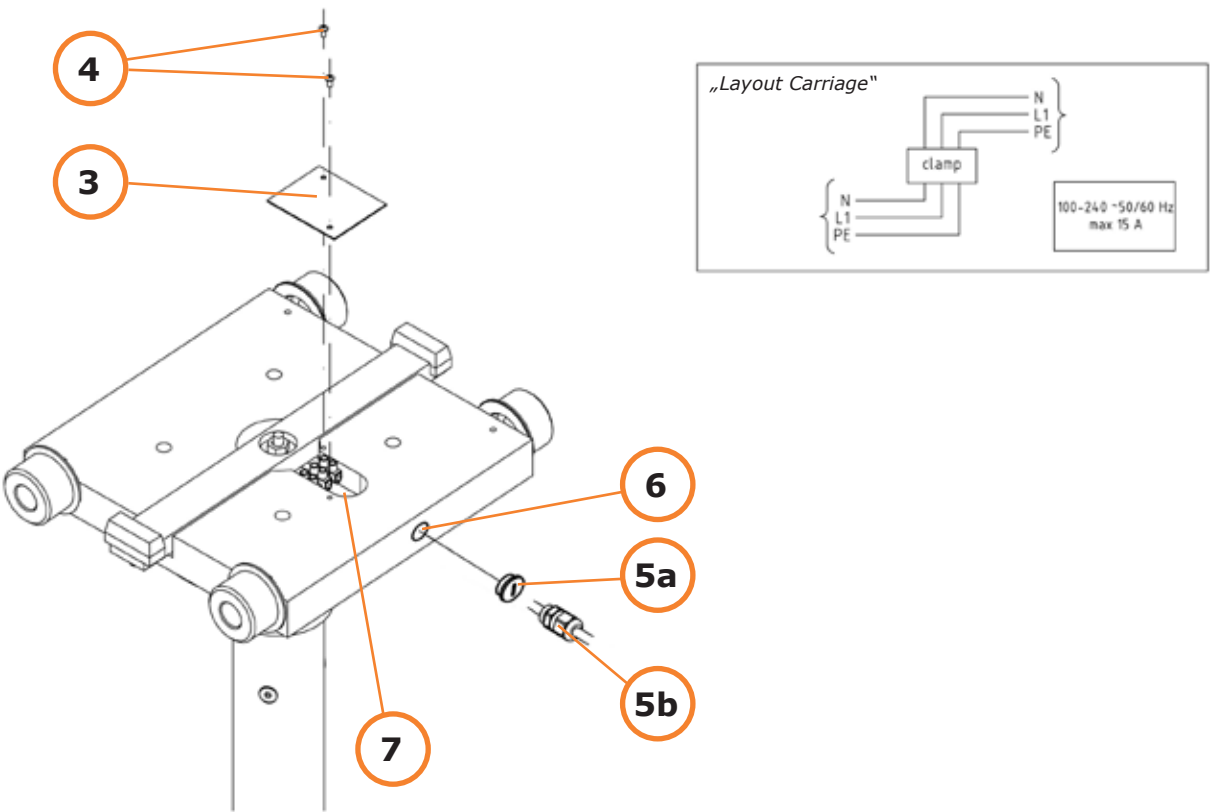
Fig. 5

4.0 Mounting the track system to the ceiling

4.2 Electrical connection of the carriage to the cable spooler

WARNING - Electric Shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

NOTE: Connect the electrical cable of the cable spooler properly to the carriage/column unit, before sliding the ceiling column with carriage into the track system. For details, see drawings (page 10).

- 4.2.1 Remove the mounted cable spooler or end plate (2/Fig.4) from the ceiling track by loosening the screws (1/Fig. 4) that secure it to the ceiling track.

WARNING - Falling Parts.

Ensure the front side cover (with cable spooler) will not fall from the ceiling track when the last screw is removed.

- 4.2.2 Remove the small plastic plate (3/Fig. 5) on the top of the carriage by removing the screws (4/Fig. 5).

NOTE: Store the small plastic plate (3/Fig. 5) and the screws (4/Fig. 5) in a safe place, for later use.

- 4.2.3 Remove the plastic cap from the carriage (5a/Fig. 5) that points toward the cable spooler from the carriage using a screwdriver.

- 4.2.4 With the help of a second person, pull the electrical cable from the cable spooler and secure it against unintentional retraction.

CAUTION - Damage to Cable Spooler.

If the electrical cable is not secured, the cable will retract and the cable drum will completely unwind itself, losing its preset spring tension. If that occurs, please remove the unit and replace it with a new cable spooler.

- 4.2.5 Push the cable through the hole (6/Fig. 5) that leads to the connection clamp on the carriage and complete the electrical connections according to „Layout Carriage“ (Fig. 5).
- 4.2.6 Secure the cable with the tension relief screwed into the carriage (7/Fig. 5).
- 4.2.7 Inspect the electrical connection to ensure its correctness.
- 4.2.8 Tighten the relief cap (5b/Fig.5) on the cable coming from the cable spooler to the carriage.
- 4.2.9 Mount the small plastic plate (3/Fig. 5) again to the top of the carriage using the screws (4/Fig. 5).
- 4.2.10 Remove the cardboard that prevents an unintentional retraction of the cable.

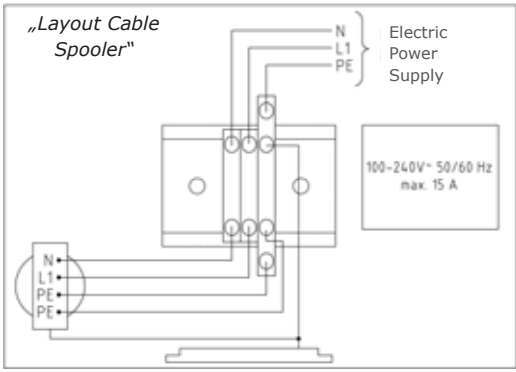
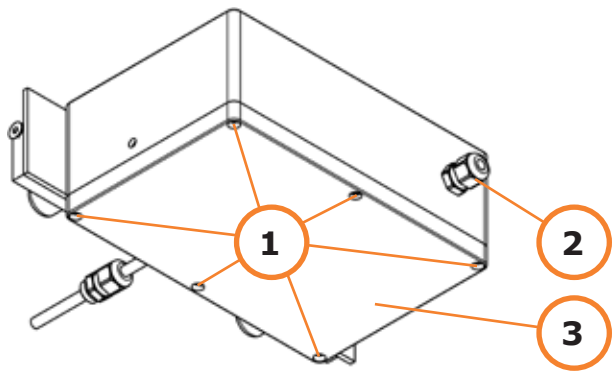
Fig. 6

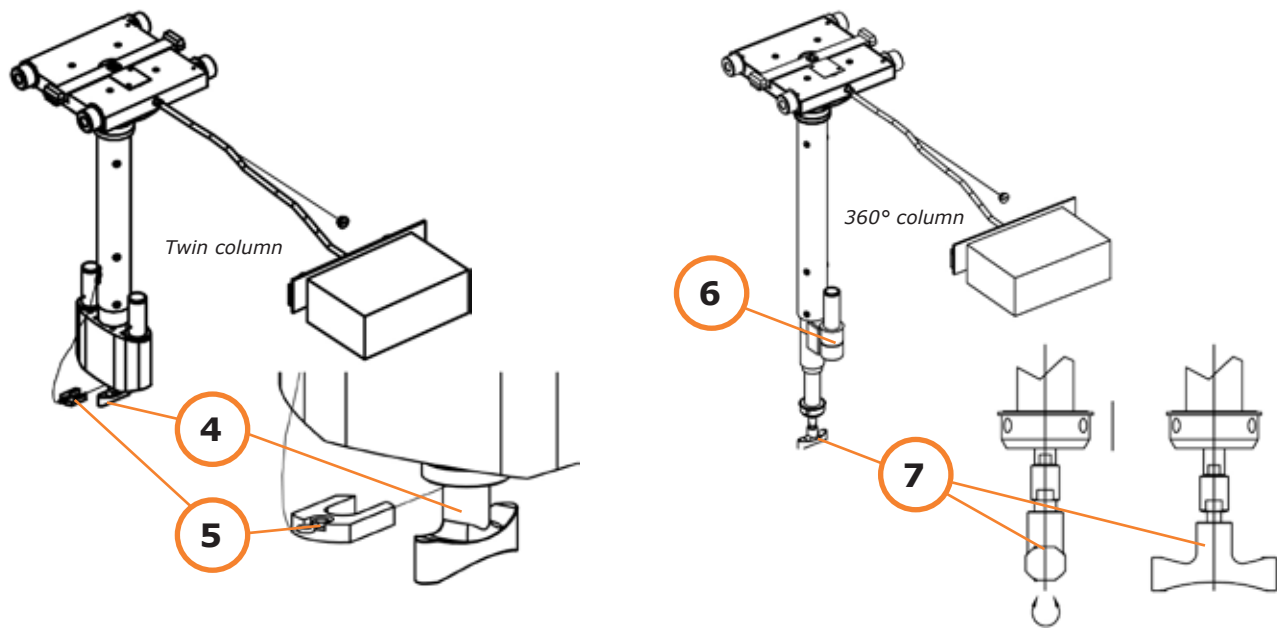
Fig. 7

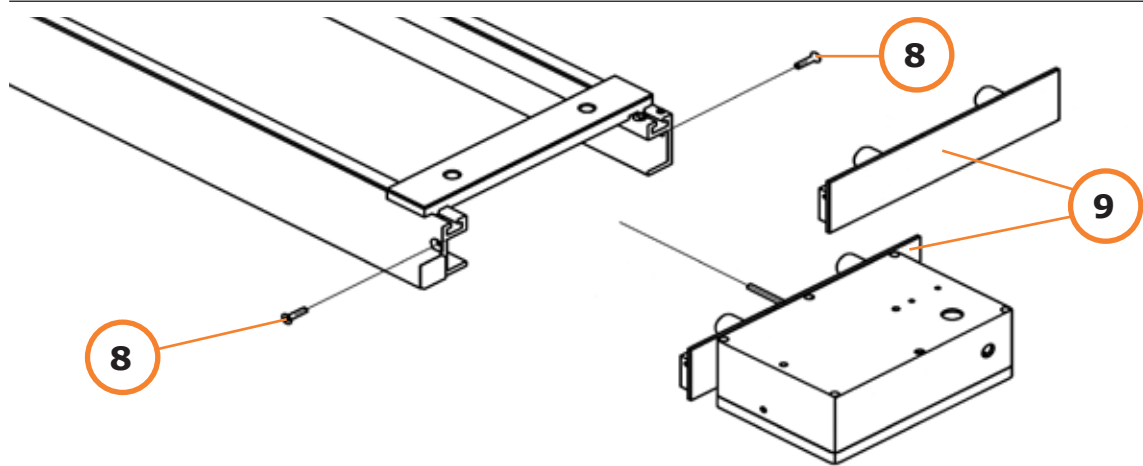
Fig. 8

4.0 Mounting the track system to the ceiling

4.3 Electrical connection of the cable spooler

- 4.3.1 Using light pressure, loosen the 6 screws (1/Fig. 6) of the cable spooler cover with a half turn.
- 4.3.2 Remove the cover (3/Fig. 6).
- 4.3.3 Guide the power cable from the outside through the provided pull relief cap (2/Fig. 6).
- 4.3.4 Complete the electrical connections according to „Layout Cable Spooler“ (Fig. 6).
- 4.3.5 Close and tighten the cable spooler cover again, and tighten the relief cap.

4.4 Mounting the carriage to the track system

- 4.4.1 If the front side cover is still mounted on the ceiling track, you can remove it by loosening the counter sunk screws (8/Fig. 8) that secure the front side cover (with/without cable spooler) to the ceiling track.
- 4.4.2 For *Twin Column* models, pull down the T-handle grip (4/Fig. 7) of the column to release the carriage brake. If desired, you can insert the attached brake release clip (5/Fig. 7).

NOTE: The position of the upper pin (6/Fig. 7) of the 360° column on SIEMENS versions may be turned 90° from what is seen in Fig. 7.

For *360° Column* models, pull down the T-handle grip (7/Fig. 7) of the column and turn it 90° to the left to release the carriage brake.

Both carriage versions can only be slid along the track when the brake is released.

- 4.4.3 Slide all wheels of the carriage into the track.
- 4.4.4 After in position on the track, lock the carriage brake again.

NOTE: After locking the carriage brake, the carriage should sit firmly on the track.

- 4.4.5 Mount the front side cover (9/Fig. 8) back to the front side of the track. Insert the counter sunk screws (8/Fig. 8) and tighten with 7 Nm torque.

Please, proceed to 7.0, „Installation/removal of plastic covers (mounting posts)“, page 22.

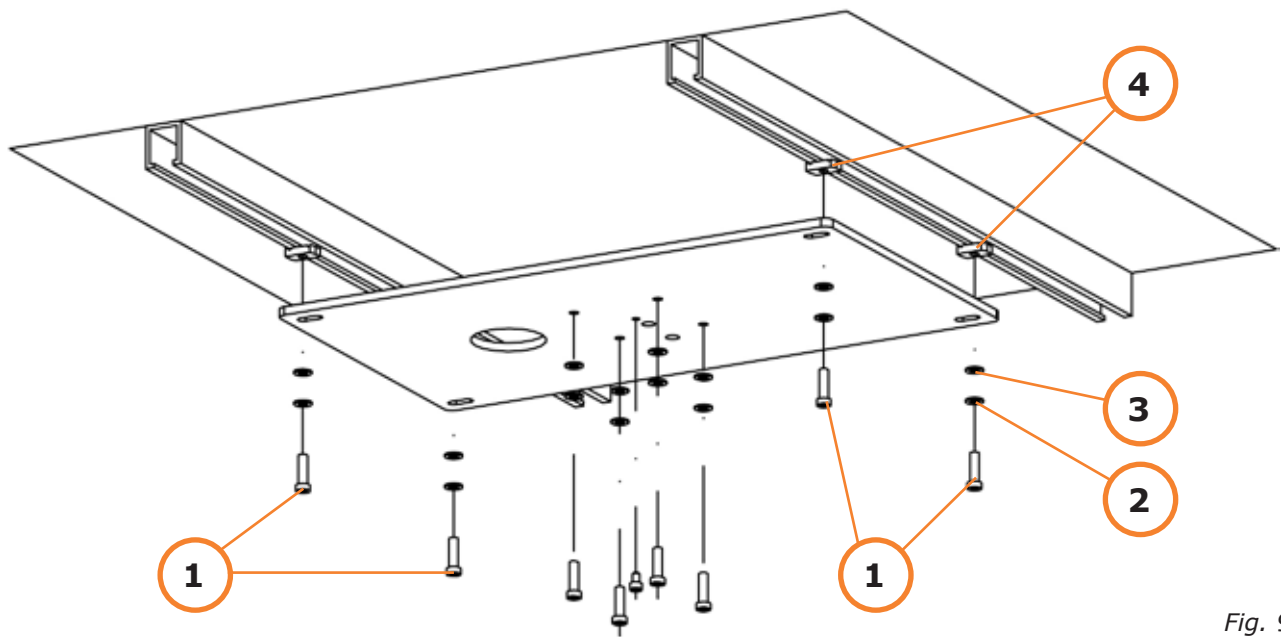
Fig. 9

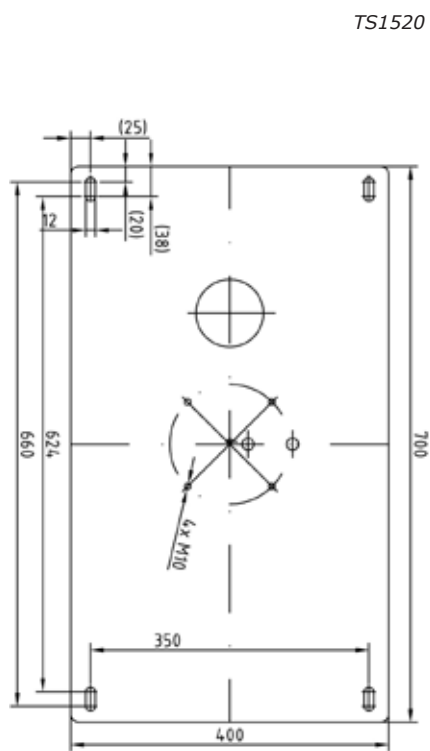
Fig. 10a


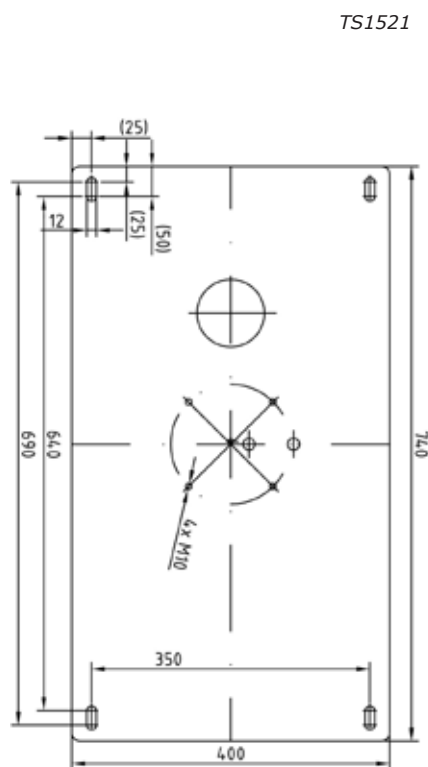
Fig. 10b

(Siemens only) Fig. 10c

5.0 Installing a Portegra2 column to a MAVIG mounting plate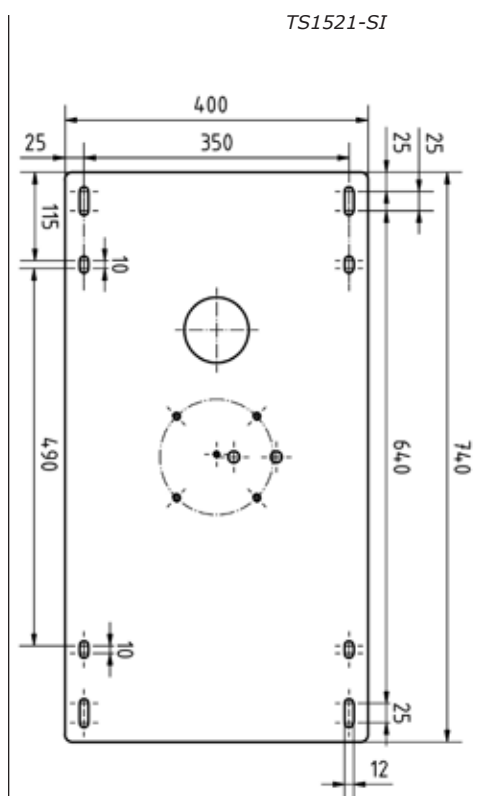

WARNING - Electric Shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

5.1 Installation of the MAVIG mounting plate to Unistrut® rails

- 5.1.1 For the correct distance between the Unistrut® rails, please see Fig. 10a, 10b and 10c.
- 5.1.2 Space the provided nuts (4/Fig. 9) along the Unistrut® rails in intervals of 350 mm (see shipping carton).
- 5.1.3 Install the mounting plate to the nuts along the Unistrut® rails using the M10x35 screws (1/Fig. 9), lock washers (2/Fig. 9), and washers (3/Fig. 9) and level with a block level, secure the screws finger-tight. If necessary, use shims.
- 5.1.4 Apply Loctite® 243 or equivalent (not included) to the screws. Tighten with 22 Nm torque.

Please, proceed to 5.3, „Installing the ceiling column to a mounting plate“.

5.2 Installation of the MAVIG mounting plate to a ceiling

Selection of appropriate installation hardware and its torque value is sole responsibility of the structural engineer. He will also define the system's layout.

- 5.2.1 The position of the mounting plate has been fixed during the project planning. When drilling holes for the mounting plate, see template Fig. 10a/10b (for a Siemens installation Fig. 10c).
- 5.2.2 Proceed with the drilling according to the template Fig. 10a/10b (for a Siemens installation Fig. 10c), and the mounting hardware requirements set by the structural engineer.
- 5.2.3 Install the mounting plate to the ceiling using the required hardware defined by the structural engineer, and level with a block level. Use shims if necessary (not included in package).
- 5.2.4 After leveling the plate, tighten the screws to the torque value specified by the structural engineer and check again the horizontal positioning with a block level.

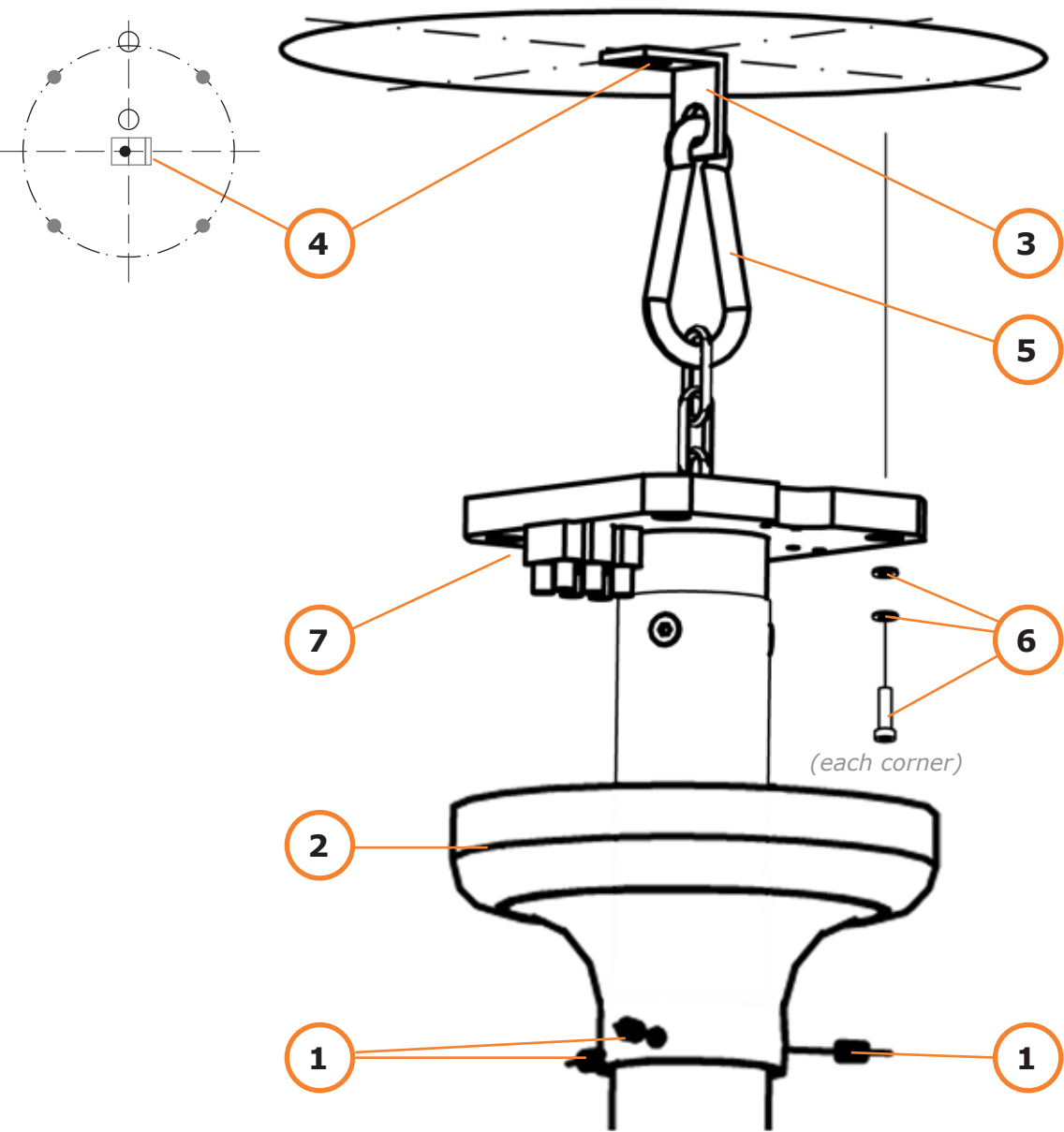
Fig. 11

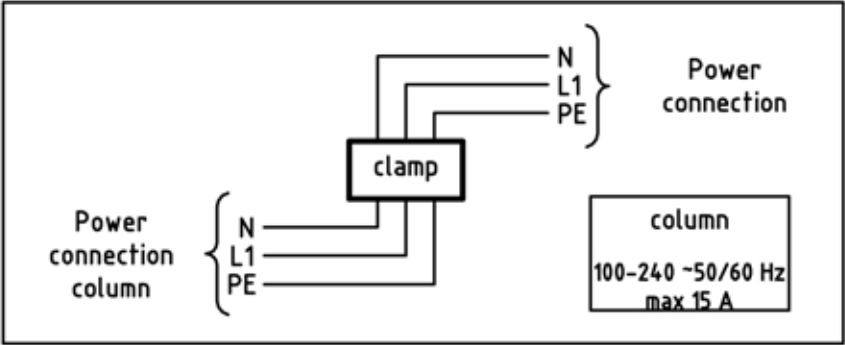
Fig. 12

5.0 Installing a Portegra2 column to a MAVIG mounting plate

5.3 Installation of the ceiling column to the mounting plate

NOTE: It is strongly recommended that at least two people perform the installation.

NOTE:  If installing the optional MAVIG isolation kit E-TS20B05, please follow the instructions included with the kit.

5.3.1 Loosen the three threaded pins (1/Fig. 11) of the canopy flange cover (2/Fig. 11).

5.3.2 Carefully, slide the canopy down the ceiling column (use caution to prevent scratches in the finish of the column), until it rests on the pins.

NOTE: To avoid damage to the finish of the ceiling column, do not secure the canopy to the column with the set screws until the installation is complete.

5.3.3 Fix the safety anchor (3/Fig. 11) to the mounting plate (4/Fig. 11) using a M8 screw and a washer.

CAUTION - Ensure Chain is Secure.

Secure the chain with the attached safety latch (5/Fig. 11) to prevent it from falling into the column tube.

5.3.4 Lift the column and engage the safety latch (5/Fig. 11) into the safety anchor (3/Fig. 11).

WARNING - Falling Parts.

After installation, ensure the safety latch is closed to ensure the column is properly secured.

5.3.5 Once the safety latch is engaged into the safety anchor, push up the ceiling column slightly and rotate the safety latch (5/Fig. 11). Make sure the small diameter of the safety latch is positioned in the safety anchor according to Fig. 11.

5.3.6 Fix the ceiling column to the mounting plate using the four M10x35 screws, lock washers, and washers (6/Fig. 11) provided, apply Loctite® 243 to the screws and tighten with 22 Nm torque.

WARNING - Improper Installation.

Only a qualified, licensed technician can perform all connections, installation, removal and repair of electrical components.

WARNING - Electric Shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

5.3.7 Ground the ceiling column with the screw marked with a grounding symbol (7/Fig. 11). Complete the electrical connection to the clamp according to Fig. 12, when electricity is needed.

5.3.8 After completing the electrical connection, check again.

5.3.9 Once the ceiling column is installed, carefully slide the canopy upward until it rests against the mounting plate and secure it with the set screws (1/Fig. 11).

Please, proceed to 7.0, „Installation/removal of plastic covers (mounting posts)“, page 22.

Drilling Template Portegra2 - Scale 1:1

The exact location of all five drill holes has to be kept, otherwise installation can't be accomplished.

A removeable copy of this template can be found on page 105 or ordered as PDF at info@mavig.com.

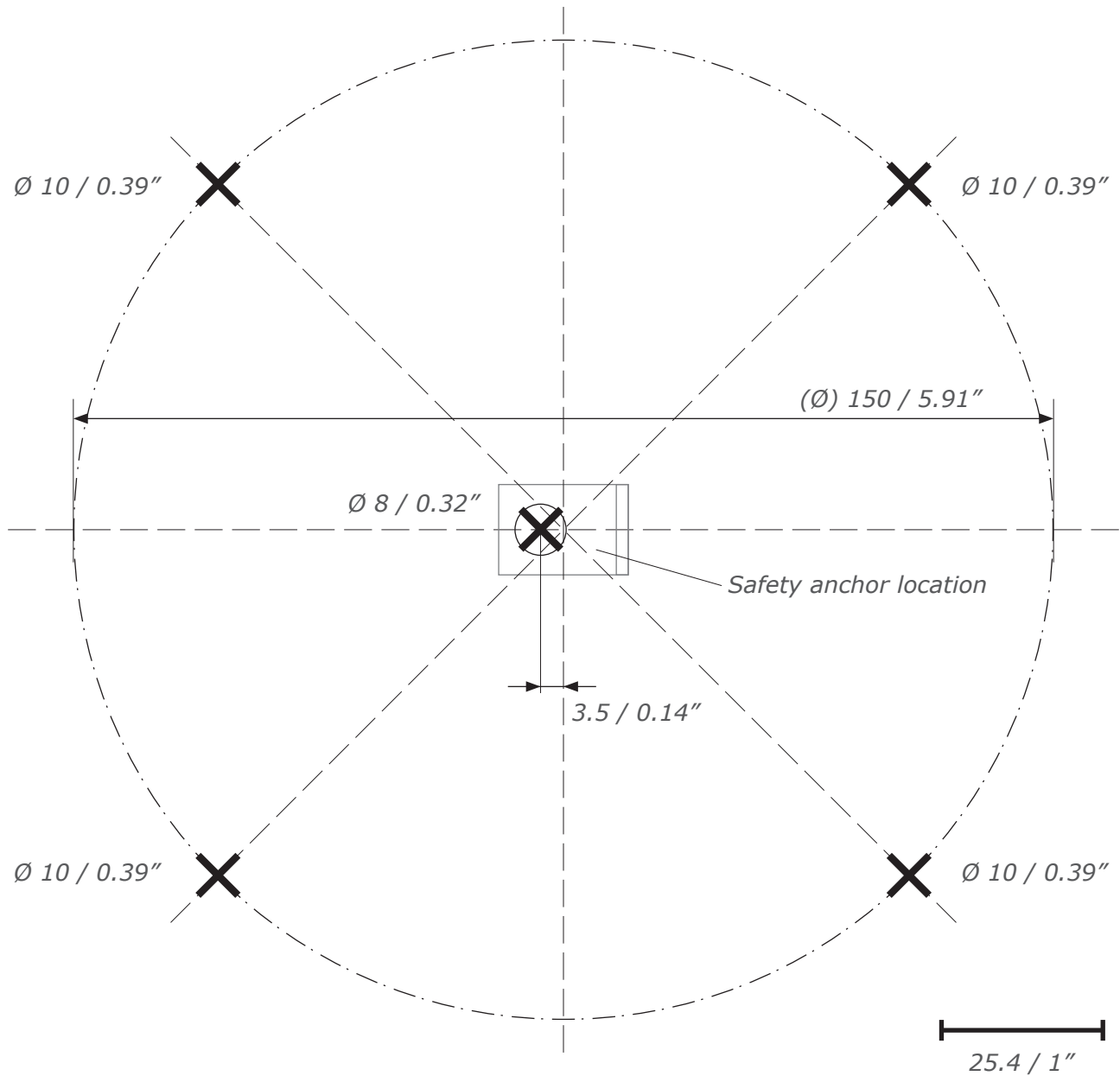
Fig. 13

6.0 Installing a Portegra2 column to a ceiling

NOTE:  If installing the optional MAVIG isolation kit E-TS20B05, please follow the instructions included with the kit.

Selection of appropriate installation hardware and its torque value is sole responsibility of the structural engineer. He will also define the system's layout.

The position of the ceiling should be determined during the project planning. For the drilling template for the flange, see Fig. 13.

Proceed with the drilling according to the template and the mounting hardware requirements set by the structural engineer.

6.1 Installation of the safety chain with safety anchor

NOTE: It is strongly recommended that at least two people perform the installation.

NOTE: The Ø 8 mm hole for the safety anchor must be 3.5 mm off center.

NOTE: Secure the chain with the attached safety latch (1/Fig. 14) to prevent it from falling into the column tube.

6.1.1 Fix the safety anchor (2/Fig. 14) to the ceiling using the installation hardware specified by the structural engineer.

6.1.2 Push up the column and engage the safety latch (1/Fig. 14) into the safety anchor (2/Fig. 14).

WARNING - Falling Parts.

After installation, ensure the safety latch is closed to ensure the column is properly secured.

6.1.3 Once the chain is engaged into the safety anchor (2/Fig. 14), push up the ceiling column slightly and rotate the safety latch (1/Fig. 14), to assure the small diameter of the safety latch being hooked into the safety anchor.

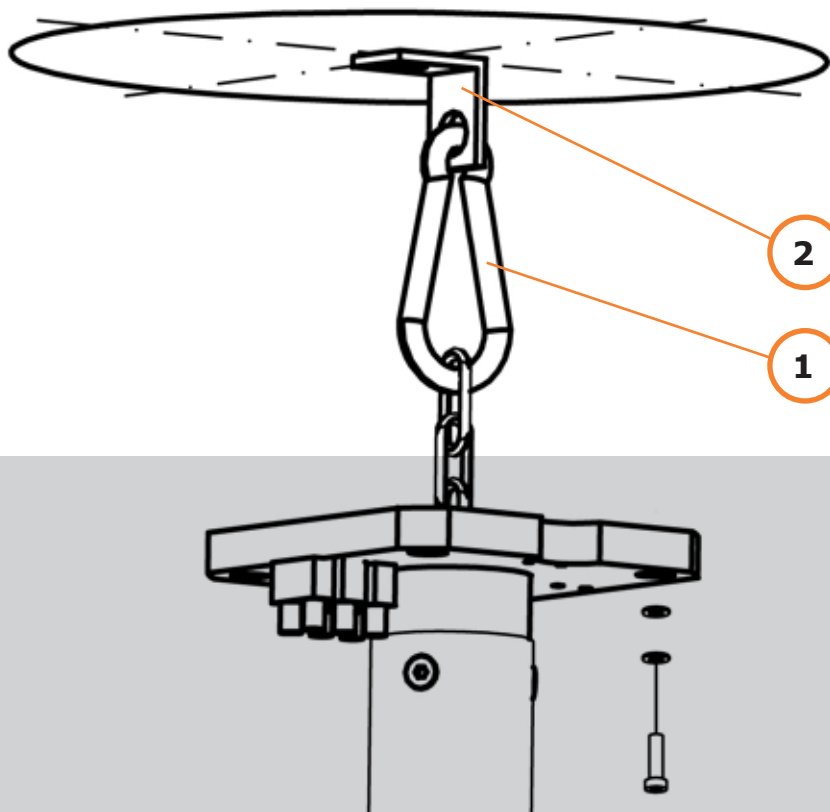
Fig. 14

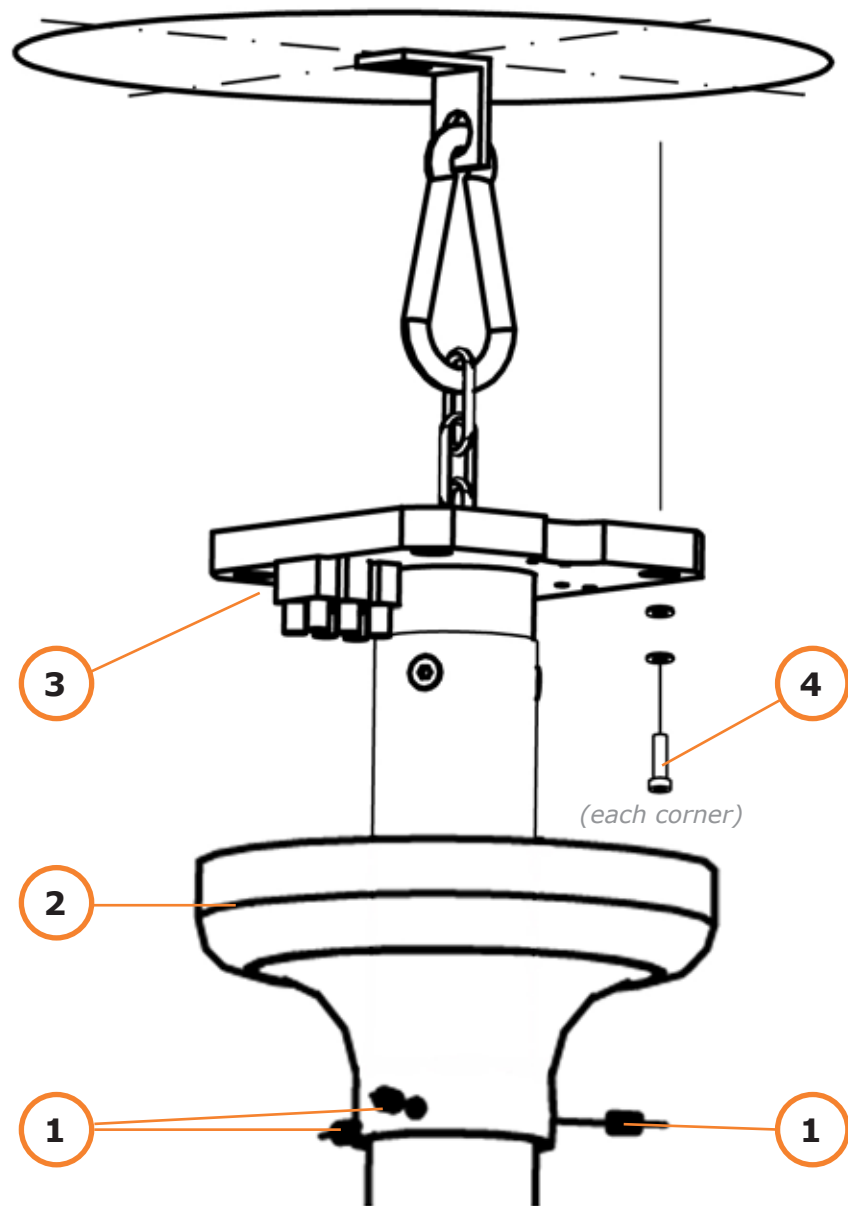
Fig. 15

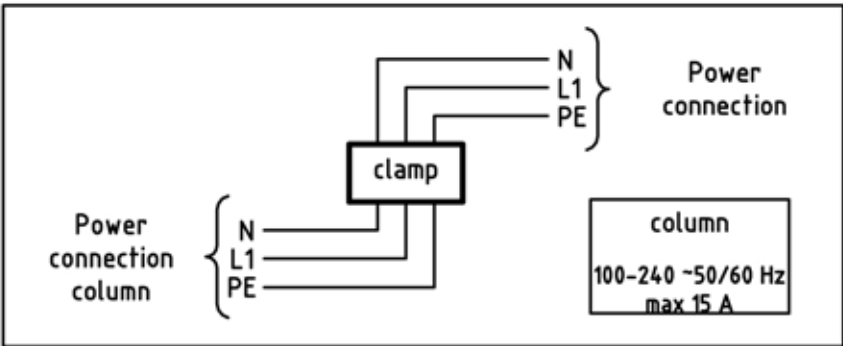
Fig. 16

6.0 Installing a Portegra2 column to a ceiling

6.2. Installation of the ceiling column to a ceiling

- 6.2.1 Loosen the three threaded pins (1/Fig. 15) of the canopy flange cover (2/Fig. 15).
- 6.2.2 Carefully, slide the canopy down the ceiling column (use caution to prevent scratches in the finish of the column) and allow it to rest on the mounting post assembly.

NOTE: To avoid damage to the finish of the ceiling column, do not secure the canopy to the column with the set screws until the installation is complete.

- 6.2.3 Fix the ceiling column to the ceiling using the installation hardware (4/Fig. 15) specified by the structural engineer.

WARNING - Electric Shock.

Electrical connections of the ceiling column must be performed by licensed or qualified personnel according to local regulations (GEF standard).

- 6.2.4 Ground the ceiling column with the screw marked with a grounding symbol (3/Fig. 15). Complete the electrical connection to the clamp according to Fig. 16, when electricity is needed.
- 6.2.5 After completing the electrical connection, check again.
- 6.2.6 Once the ceiling column is installed, carefully slide the canopy upward until it rests against the ceiling and secure it with the set screws (1/Fig. 15).

Please, proceed to 7.0, „Installation/removal of plastic covers (mounting posts)“, page 22.

360° column

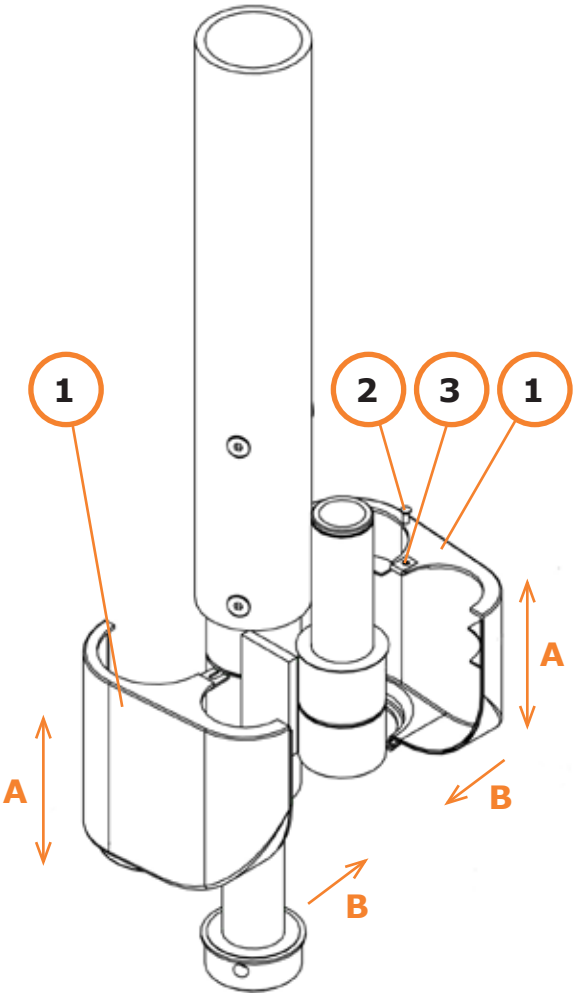
Fig. 17

Twin column

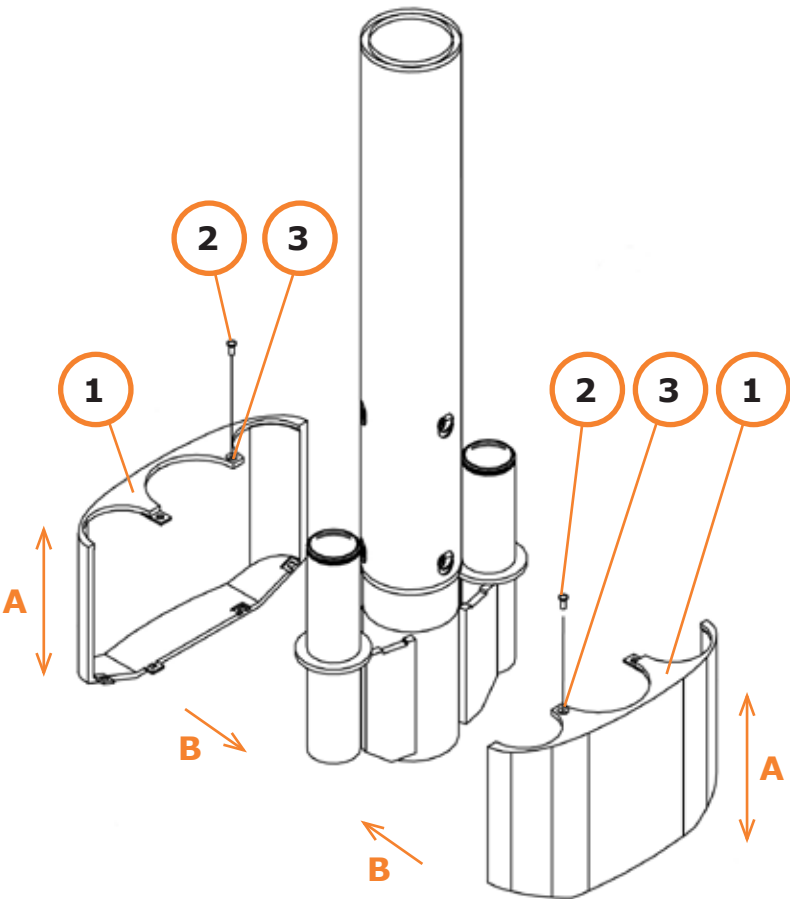
Fig. 18

7.0 Installation/removal of plastic covers (mounting posts)

To avoid damage to the plastic cover during shipping, the two halves comprising the cover (1/Fig. 17/18) must be attached on site. To ensure proper installation of the cover, please complete following steps:

7.1 Installation

- 7.1.1 Remove the plastic cover from the shipping carton; ensure it is in good condition. Carefully remove the mounting screw(s) (2/Fig. 17/18) and speed nut(s) (3/Fig. 17/18) making sure to store them in a safe place for later use.
- 7.1.2 Place the speed nut(s) (3/Fig. 17/18) on screw hole of the column cover.
- 7.1.3 Ensure both halves vertically align (A/Fig. 17/18). Once the height of both halves is level, gently press them in direction (B/Fig. 17/18). Ensure the pegs inside the cover will click into the grooves when snapping them together.

CAUTION - Snaps can break off.

When mounting or removing covers again, be very careful as the snaps can break off. This can cause the covers not to stay in place, and can fall on patient during procedures.

CAUTION - Falling parts.

Make sure that the cover's screws (2/Fig. 17/18) are not screwed too tight, stripping the thread of the cover. Replace the cover if the screw hole has been stripped.

- 7.1.4 To secure the plastic cover to the column, tighten the mounting screw (2/Fig. 17/18) into the thread.

7.2 Removal

NOTE: The disassembly should be carried out without applying much force.

- 7.2.1 Remove the screw(s) (2/Fig. 17/18) from the plastic cover and store them in a safe place.
- 7.2.2 Gently, separate the plastic halves near the top, in direction opposite to (B/Fig. 17/18).
- 7.2.3 Separate the snaps from the grooves with the help of a screwdriver.
- 7.2.4 Remove the plastic halves completely.
- 7.2.5 After the removal of the plastic halves, insert the screw(s) (2/Fig. 17/18) into the thread to avoid losing it.

Please proceed to 9, „Installation of the extension/spring arm combination“, page 26.

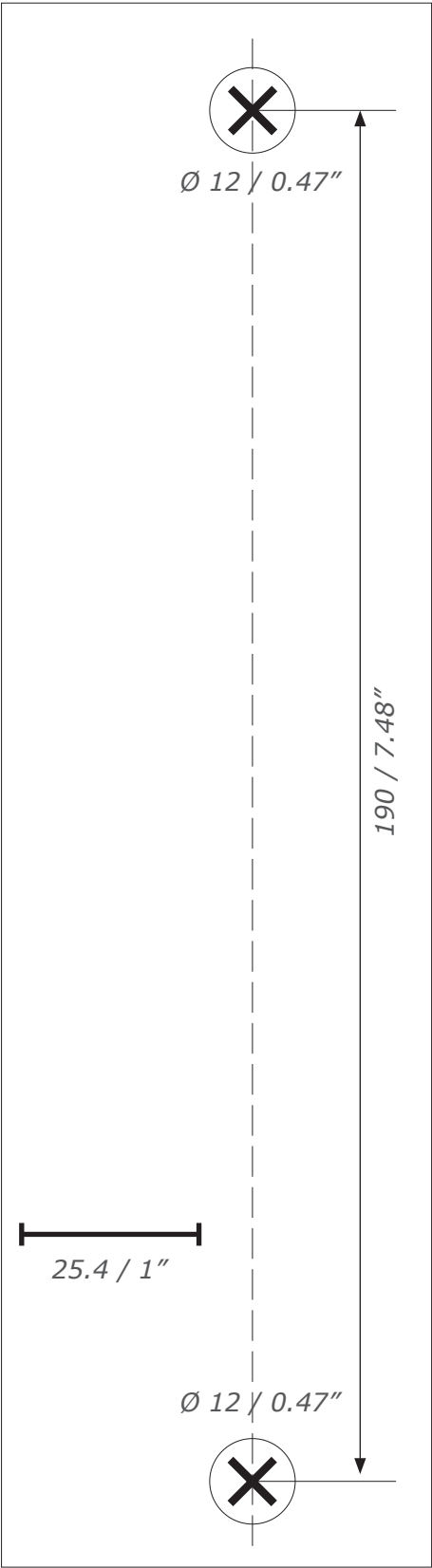
Fig. 19a

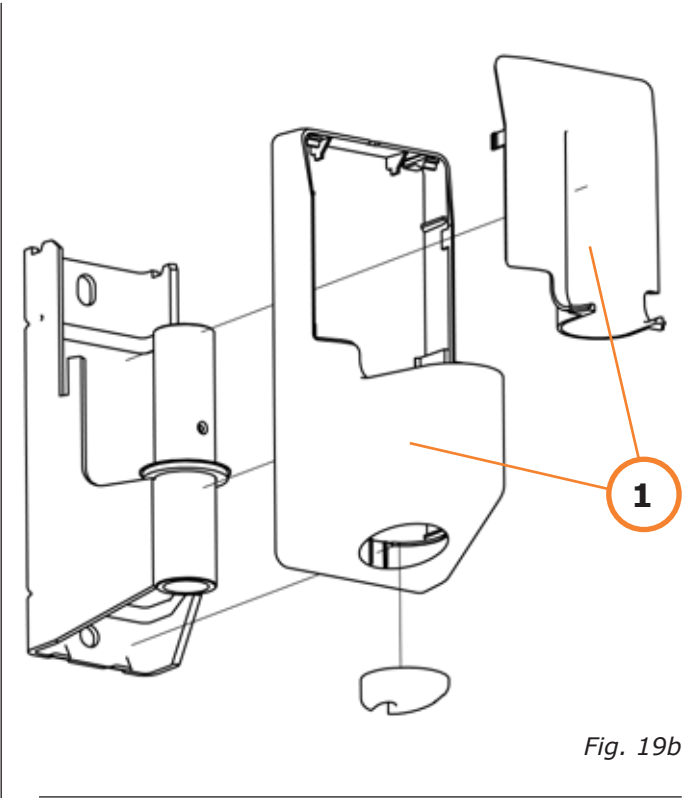
Fig. 19b

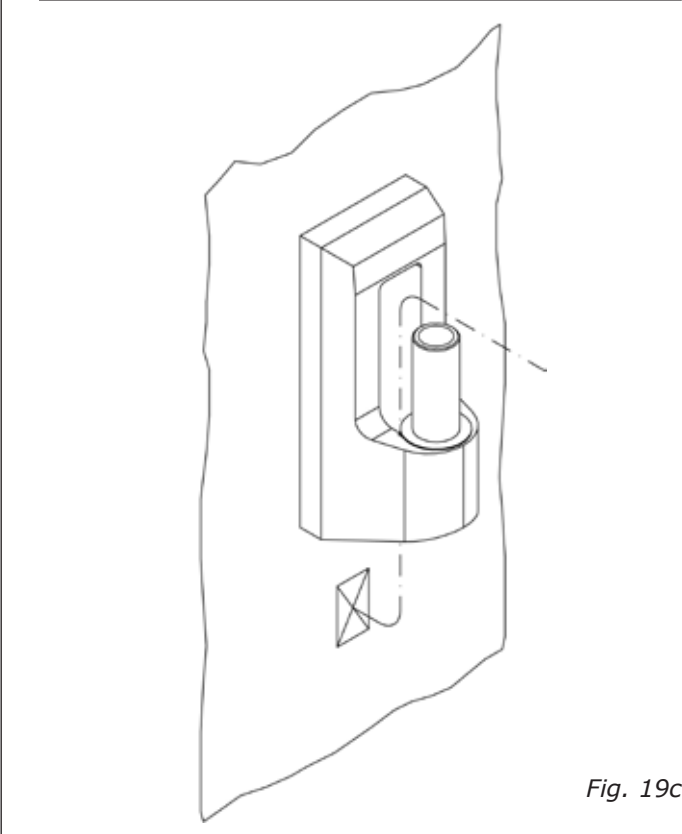
Fig. 19c

8.0 Installation of the wall mount bracket

8.1 Installing the wall mount bracket

- 8.1.1 Selection of appropriate installation hardware and its torque value is sole responsibility of the structural engineer. He may also select and specify further installing steps.
- 8.1.2 Position the mounting assembly as planned. For the distance between the drill holes see the drilling template (Fig. 19a).
- 8.1.3 Drill the mounting holes for the mounting hardware as defined and selected by the structural engineer.
- 8.1.4 To ensure the wall mount bracket will be solidly installed, please refer to local regulations.
- 8.1.5 Level the mounting pin with a block level.
- 8.1.6 Fix the wall mount bracket to the wall with the mounting hardware specified by the structural engineer.
- 8.1.7 Mount the plastic covers (1/ Fig. 19b) by clicking them in place as shown in Fig. 19b.

8.2 Electrical connection

- 8.2.1 The wall-mounted arm is equipped with a three way connector. For electrical connections we recommend the installation of a wall-socket next to the wall mount bracket. (Fig. 19c)

WARNING - Electrical Connections.

Electrical connections of the wall mount bracket must be performed by licensed or qualified personnel according to local regulations (GEF standard).


WARNING - Electric Shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

- 8.2.2 After completing the electrical connection, check again.

360° column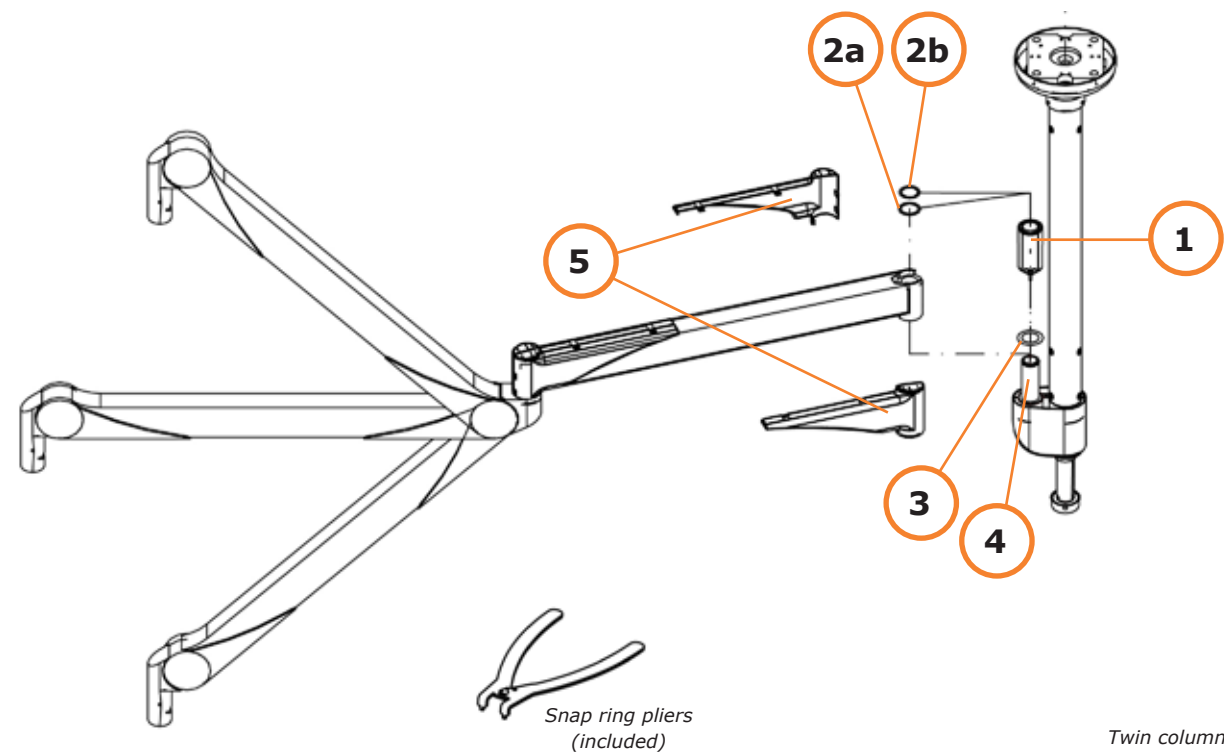

Twin column

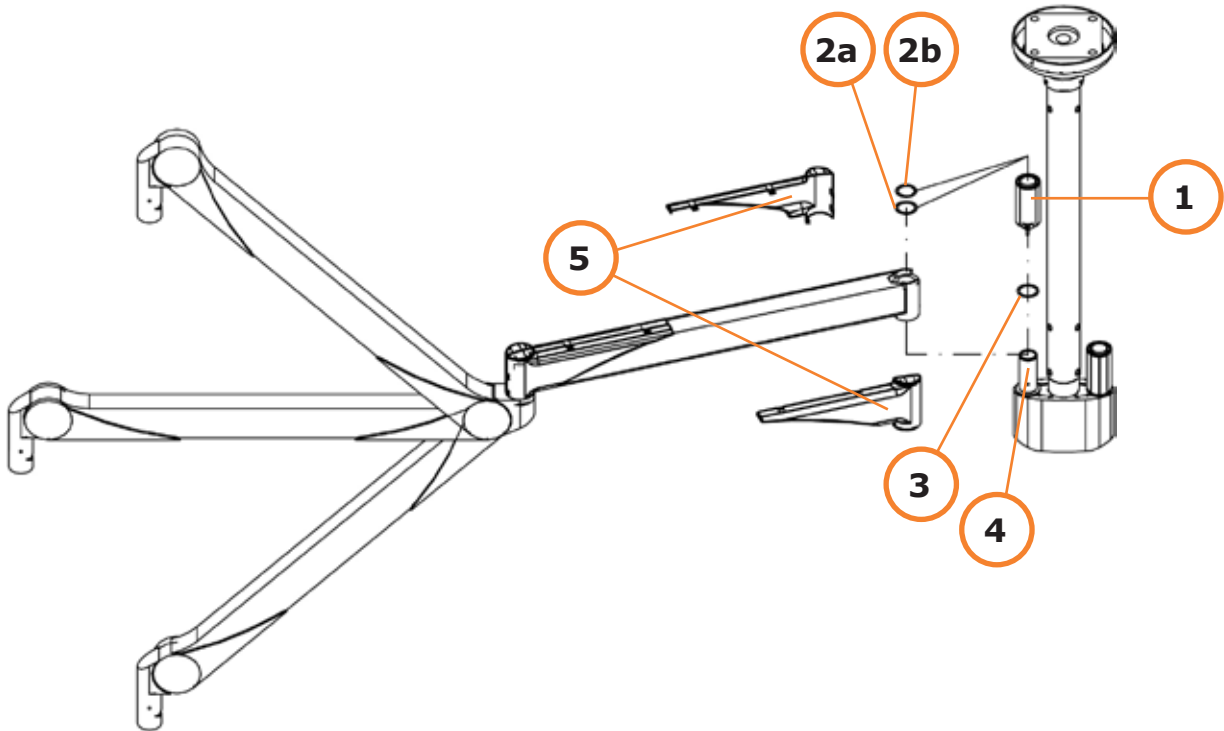
Fig. 20

9.0 Installation of the extension/spring arm combination

9.1 Installation of the extension/spring arm combination to the upper post

NOTE: It is strongly recommended that at least two people perform the installation.

WARNING - Electric Shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

CAUTION - Incorrect Mounting Post.

If an electrical extension/spring arm combination is required, it must be mounted to the electric mounting post. Only one mounting post is electrical, and can be identified by the female socket that can be seen when looking down from the top of the mounting post.

9.1.1 Loosen the screw and remove the plastic cap of the pin (1/Fig. 20).

NOTE: You will find a spacer (2a/Fig. 20) and a snap ring (2b/Fig. 20) on top of the plastic cap of the mounting pin. They will be used later during installation. Do not throw away.

9.1.2 Ensure the washer (3/Fig. 20) is positioned on the contact surface of the mounting pin, below the arm. Please, inform your supplier in case it should be missing, since it is necessary for proper operation.

9.1.3 Slide the extension/spring arm combination onto the mounting pin (4/Fig. 20) of the ceiling column.

9.1.4. If the extension/spring arm is electrical, pull out the male socket plug (pin side) from the bearing of the extension arm. Disconnect the plug, in order to be able to place the spacer and snap ring onto the extension arm, once the extension arm has been placed onto the pin firmly.

9.1.5. Remove the spacer (2a/Fig. 20) and the snap ring (2b/Fig. 20) from the top of the plastic cover (1/Fig. 20), and put them in place onto the mounting pin above the extension arm, placing the spacer first (2a/Fig. 20) and then the snap ring on top (2b/Fig. 20).

WARNING - Using Improper Tools.

For a safe installation, ensure the snap ring is fully placed in the groove on all sides. For the snap ring installation use only the tool included. **Do not use other tools for this purpose.** If the snap ring is not properly installed, the arm assembly may work loose, resulting in operator or patient injury.

9.1.6 If the extension/spring arm is electrical, insert the male socket plug into the female socket receptacle on the mounting post joint.

9.1.7 Attach the covers (5/Fig. 20) of the extension/spring arm, securing them with the mounting screws. Ensure the covers sit firmly.

CAUTION - Falling parts.

Make sure that the cover's screws are not screwed too tight, stripping the thread of the cover. Replace the cover if the screw hole has been stripped.

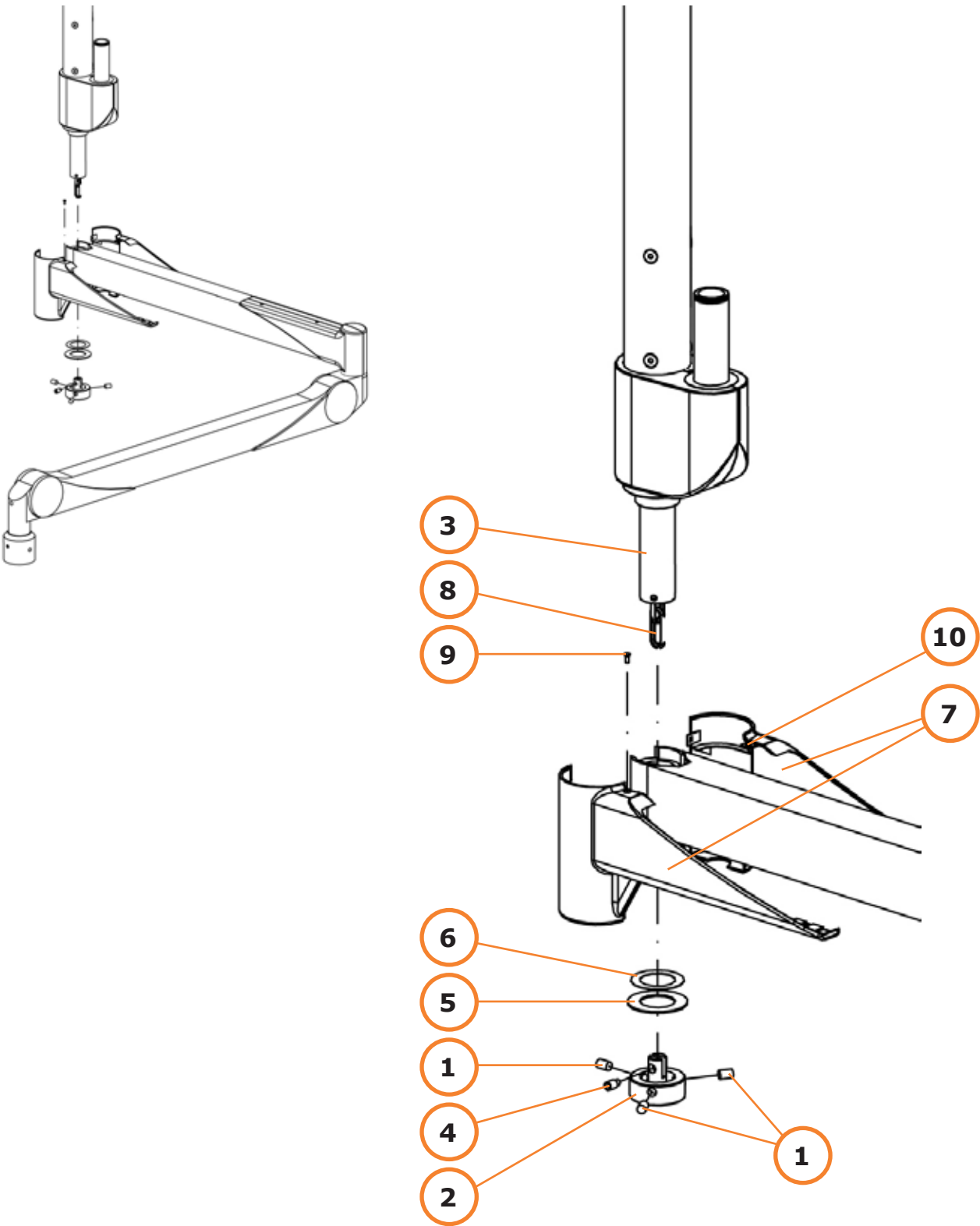
Fig. 21

9.0 Installation of the extension/spring arm combination

9.2 Installation of the extension/spring arm combination to the lower post of a 360° column - stationary version

NOTE: It is strongly recommended that at least two people perform the installation.

CAUTION - Improper Mounting Post.

Only a non-electrical extension/spring arm combination can be mounted to the lower mounting post.

- 9.2.1 Loosen the 3 set screws (1/Fig. 21) and carefully slide the locking ring (2/Fig. 21) off the mounting post (3/Fig. 21).
- 9.2.2 Remove the lock screw (4/Fig. 21) from the locking ring (2/Fig. 21).
- 9.2.3 Remove the protective cap from the mounting post (3/Fig. 21).
Store screws (1,4/Fig. 21), locking ring (2/Fig. 21) and both washers (5,6/Fig. 21) for re-use.

NOTE: If the two plastic halves of the arm cover (7/Fig. 21) are already connected, gently separate them.

- 9.2.4 Mount the extension/spring arm combination firmly onto the 360° mounting post (3/Fig. 21).
- 9.2.5 Place first the metal washer (6/Fig. 21) and then the plastic washer (5/Fig. 21) under the extension arm, onto the 360° mounting post (3/Fig. 21).
- 9.2.6 Connect the end of the safety chain (8/Fig. 21) with the locking ring (2/Fig. 21).
- 9.2.7 Apply Loctite® 270 to the lock screw (4/Fig. 21) and tighten it into the locking ring (2/Fig. 21) with 20 Nm torque.
- 9.2.8 Mount the locking ring (2/Fig. 21) with the two washers (5,6/Fig. 21) on the 360° mounting post.
- 9.2.9 Apply Loctite® 270 to the 3 set screws (1/Fig. 21) and secure the locking ring (2/Fig. 21) to the mounting post (3/Fig. 14) with them using 20 Nm torque.
- 9.2.10 Remove the arm cover (7/Fig. 21) from the ceiling column shipping carton. Use this instead of the arm cover supplied with the Portegra2 extension/suspension.
- 9.2.11 Place the speed nut (10/Fig. 21) on the screw hole tab of the the arm cover. Attach the two halves of the cover underneath the locking ring (2/Fig. 21); carefully fit them together and click the snaps into the grooves.

CAUTION - Snaps can break off.

When mounting or removing covers again, be very careful as the snaps can break off. This can cause the covers not to stay in place, and can fall on patient during procedures.

CAUTION - Falling parts.

Make sure that the cover's screws (9/Fig. 21) are not screwed too tight, stripping the thread of the cover. Replace the cover if the screw hole has been stripped.

- 9.2.14 Insert and tighten the mounting screw (9/Fig. 21) into the thread provided.

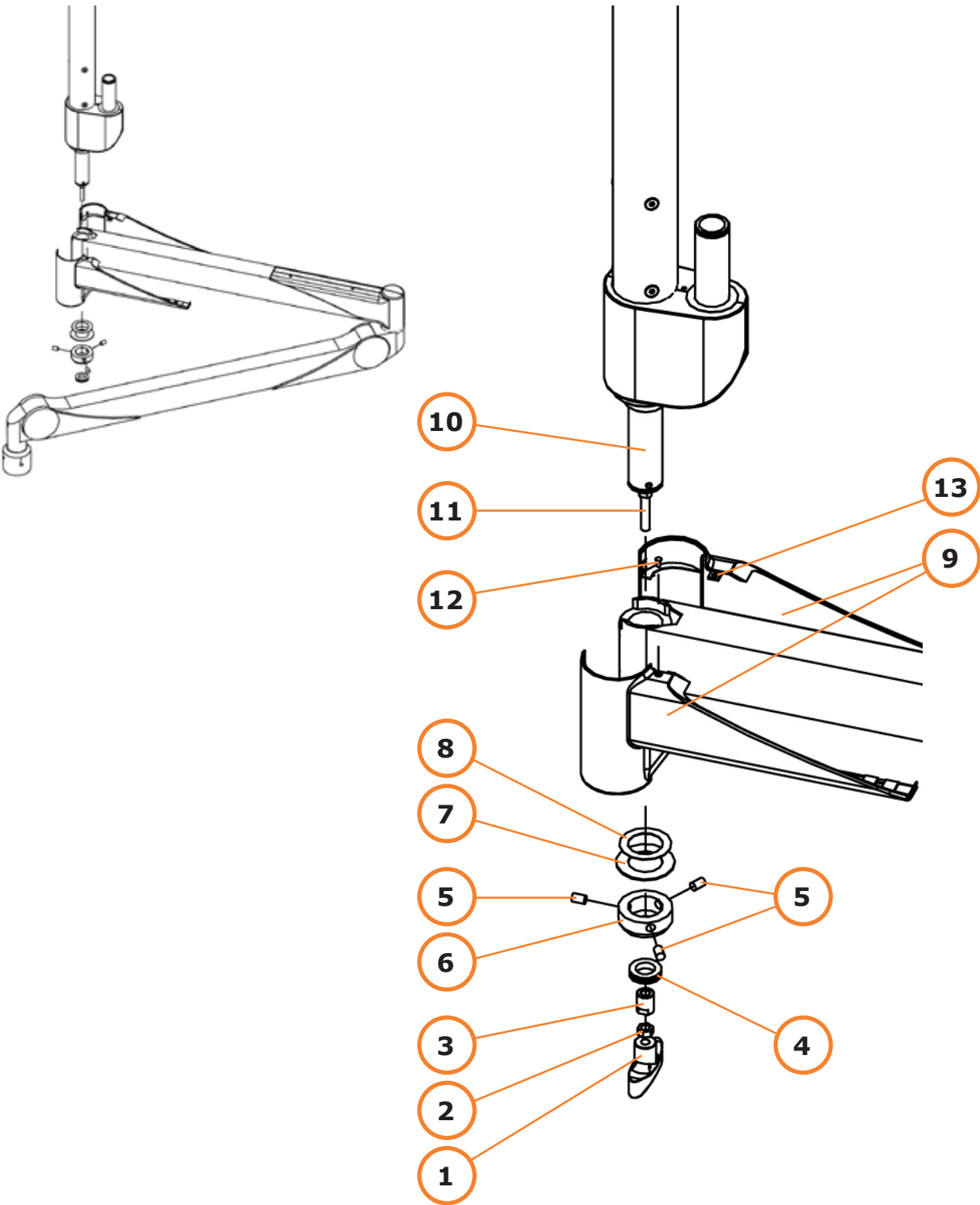
Fig. 22

9.0 Installation of the extension/spring arm combination

9.3 Installation of the extension/spring arm combination to the lower post of a 360° column - carriage/track version

NOTE: It is strongly recommended that at least two people perform the installation.

CAUTION - Incorrect Mounting Post.

Only a non-electrical extension/spring arm combination can be mounted to the lower mounting post.

9.3.1 Remove/unscrew the following elements from the 360° mounting pin.

- | | | | | | |
|----|---------------|-------------|----|--------------|-------------|
| 1. | T-handle grip | (1/Fig. 22) | 3. | Locking ring | (3/Fig. 22) |
| 2. | Nut | (2/Fig. 22) | 4. | Cover ring | (4/Fig. 22) |

9.3.2 Remove all 3 screws (5/Fig. 22) in order to carefully remove the safety ring (6/Fig. 22) and both washers (7,8/Fig. 22) from the mounting pin (10/Fig. 22).

9.3.3 Carefully, remove the protection cap from the mounting pin (10/Fig. 22).

9.3.4 Mount the extension/spring arm combination onto the 360° mounting pin (10/Fig. 22).

WARNING - Incorrect Mounting Order.

Ensure the below assembly is properly mounted in the correct order. A structural failure of the arm assembly could result in operator or patient injury.

9.3.5 Place first the metal washer (8/Fig. 22) and then the plastic washer (7/Fig. 22) on the 360° mounting pin.

9.3.6 Apply Loctite® 270 to the 3 set screws (5/Fig. 22) and secure the safety ring (6/Fig. 22) onto the 360° mounting pin with them. Tighten with 20 Nm torque.

9.3.7 Tighten first the locking ring (3/Fig. 22) and then the hexagon nut (2/Fig. 22) to the switch cam.

9.3.8 Slide the cover ring (4/Fig. 22) onto the switch cam (11/Fig. 22).

9.3.9 Apply Loctite® 243 to the T-handle grip (1/Fig. 22) and tighten it with the nut (2/Fig. 22) to the switch cam (11/Fig. 22).

9.3.10 Remove the arm cover (9/Fig. 22) from the ceiling column shipping carton. Use this cover instead of the arm cover supplied with the Portegra2 extension/suspension.

9.3.11 Place the speed nut (13/Fig. 22) on the screw hole tab of the the arm cover. Attach the two halves of the cover (9/Fig. 22) to the arm, ensuring that the cover ring (4/Fig. 22) fits into the hole cut-out of the bottom of the covers. Carefully fit them together and click the pegs into the grooves.

CAUTION - Falling parts.

Make sure that the cover's screws (12/Fig. 22). are not screwed too tight, stripping the thread of the cover. Replace the cover if the screw hole has been stripped.

9.3.12 Insert and tighten the mounting screw (12/Fig. 22) into the thread provided.

Portegra2 Spring Arm End Positions

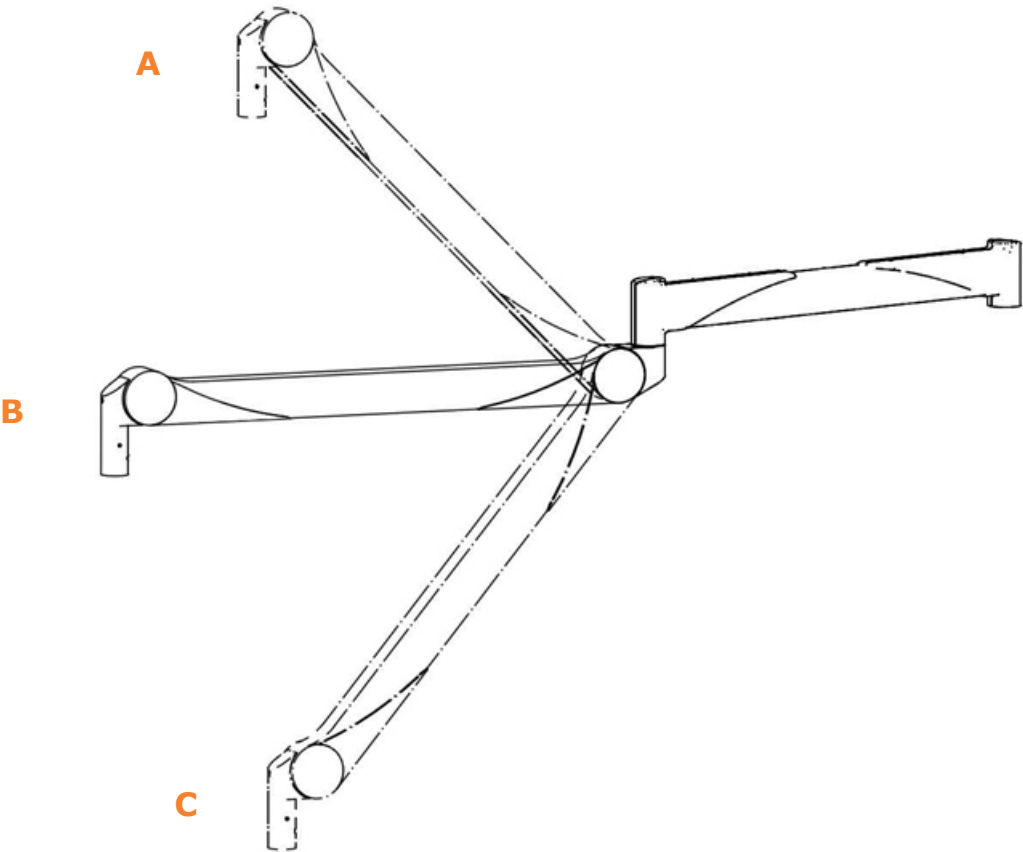
Fig. 23

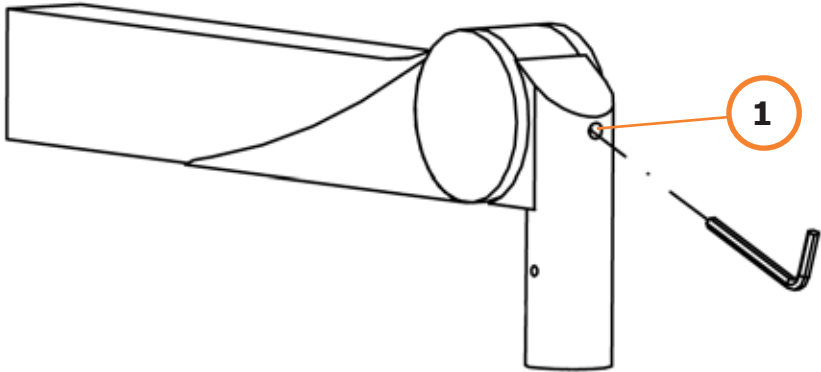
Fig. 24

10.0 Installation and removal of adaptable elements

WARNING - Improper Installation.

All components must be installed or removed as described. The loosening of any screws that are not mentioned in this manual is prohibited, as they are secured with an appropriate thread locking adhesive and fixed with the appropriate torque. Unauthorized loosening of screws can weaken the system and result in damage or injury.

NOTE: The Portegra2 systems are built and validated for MAVIG original devices. Only these MAVIG devices are optimized and equipped with all safety features. MAVIG cannot accept any warranty or liability for non-MAVIG devices attached to Portegra2 systems, unless MAVIG has validated and approved such devices.

CAUTION - Special Steps.

The installation process described below is the standard process. Before installing an end device please refer to the device's installation manual or instructions for use for any special conditions or steps needed.

10.1 Installation of non-electrical or externally cabled end devices

NOTE: It is strongly recommended that at least two people perform the installation.

WARNING - Non electrical.

Ensure the device will be mounted to a *non-electrical* spring arm.

WARNING - Under Constant Spring Tension.

Set the upper stop of the spring arm to the horizontal position (Position B/ Fig. 23) during the installation process. To correct this, insert a 5 mm allen wrench into the opening (1/ Fig. 24) and turn it fully to the right, otherwise the spring arm will uncontrollably swing upwards.

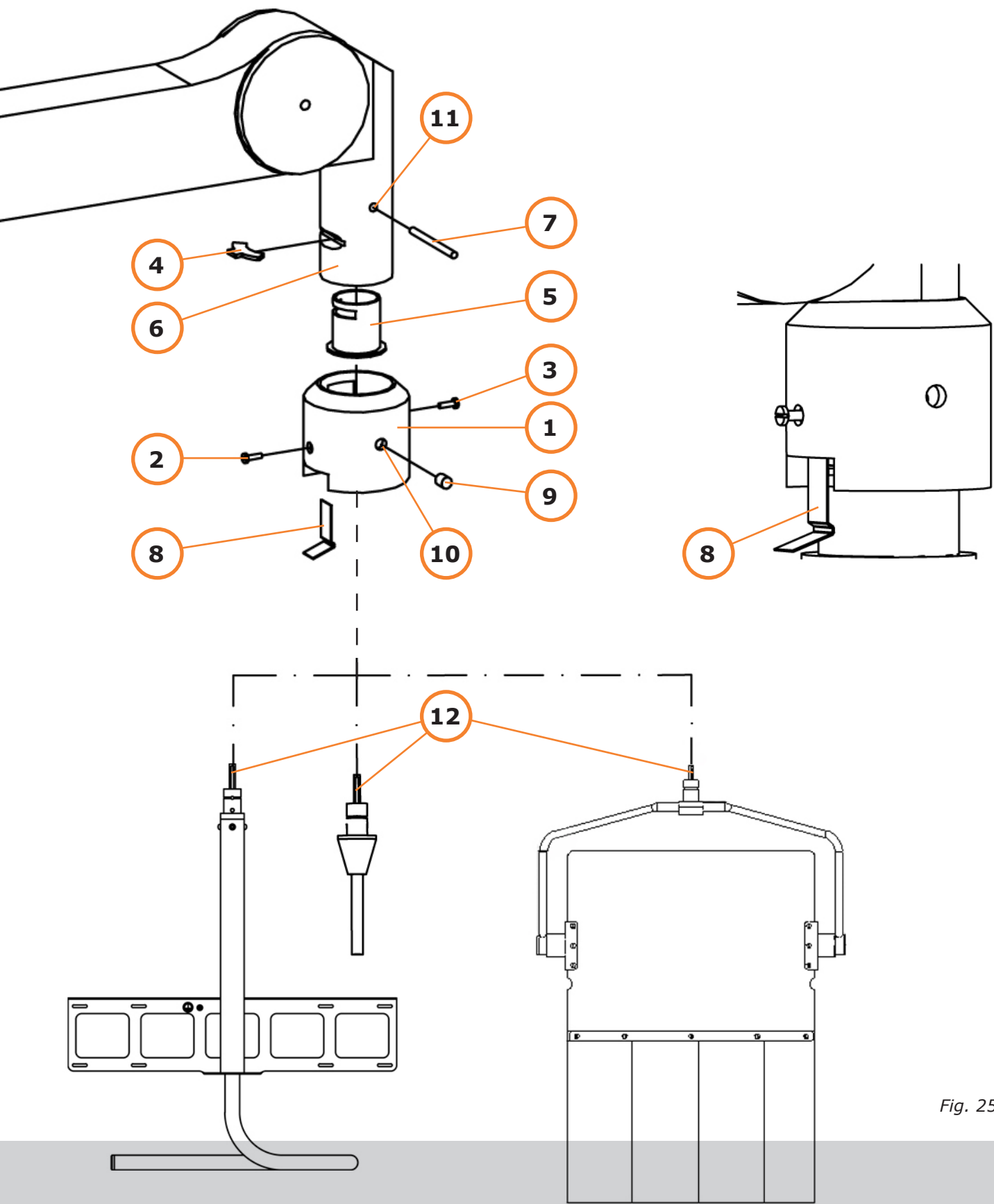
Fig. 25

10.0 Installation and removal of adaptable elements

10.1 Installation of non-electrical or externally cabled end devices

10.1.1 Remove the screws (2,3/Fig. 25) from the safety collar (1/Fig. 25). Store them in a safe place.

NOTE: It is best to use a screwdriver with a magnetic tip to loosen the screws.

10.1.2 Push up the safety collar (1/Fig. 25) until it rests on the plastic cover and hold it in this position.

CAUTION - Falling Parts.

Hold the safety collar (1/Fig. 25) to ensure that it will not fall off the spring arm.

10.1.3 Remove the retaining clip (4/Fig. 25) from its slot Fig. 25 and remove the protective cap (5/Fig. 25) from the receptacle (6/Fig. 25).

NOTE: The eyehole of the safety pin/connecting part (12/Fig. 25) must line up with the drill holes (10,11/Fig. 25) to allow the insertion of the straight pin (7/Fig. 25).

10.1.4 Insert the safety pin/connecting part of the adaptable element (12/Fig. 25) into the receptacle (6/Fig. 25) until it rests on its flat surface. Insert the retaining clip (4/Fig. 25) in its slot.

10.1.5 Position the safety collar (1/Fig. 25) until the drill holes (10/Fig. 25) align with the holes (11/Fig. 25) in the receptacle.

10.1.6 Insert the straight pin enclosed (7/Fig. 25) into the drill hole. If correctly installed, the safety collar (1/Fig. 25) can now slide freely downward.

CAUTION - Long End First.

Remove the safety spring (8/Fig. 25) from your shipping carton and insert it, long end first, into the safety collar (1/Fig. 25) as shown in detail (Fig. 25).

NOTE: Always use a new MAVIG safety spring. You will find a replacement spring in the shipping carton.

10.1.7 Secure the safety spring (8/Fig. 25) under the retaining clip (4/Fig. 25).

10.1.8 Carefully, slide the safety collar (1/Fig. 25) over the safety spring (8/Fig. 25), until it is flush with the flat surface of the receptacle. While sliding down the safety collar, make sure the safety spring is firmly set.

10.1.9 Insert and tighten the screws (2,3/Fig. 25) into the drill holes as shown in the figure.

10.1.10 Remove the plastic plugs (9/Fig. 25) from your shipping carton and insert them into the drill holes (10/Fig. 25) on the sides of the safety collar (1/Fig. 25).

Portegra2 Spring Arm End Positions

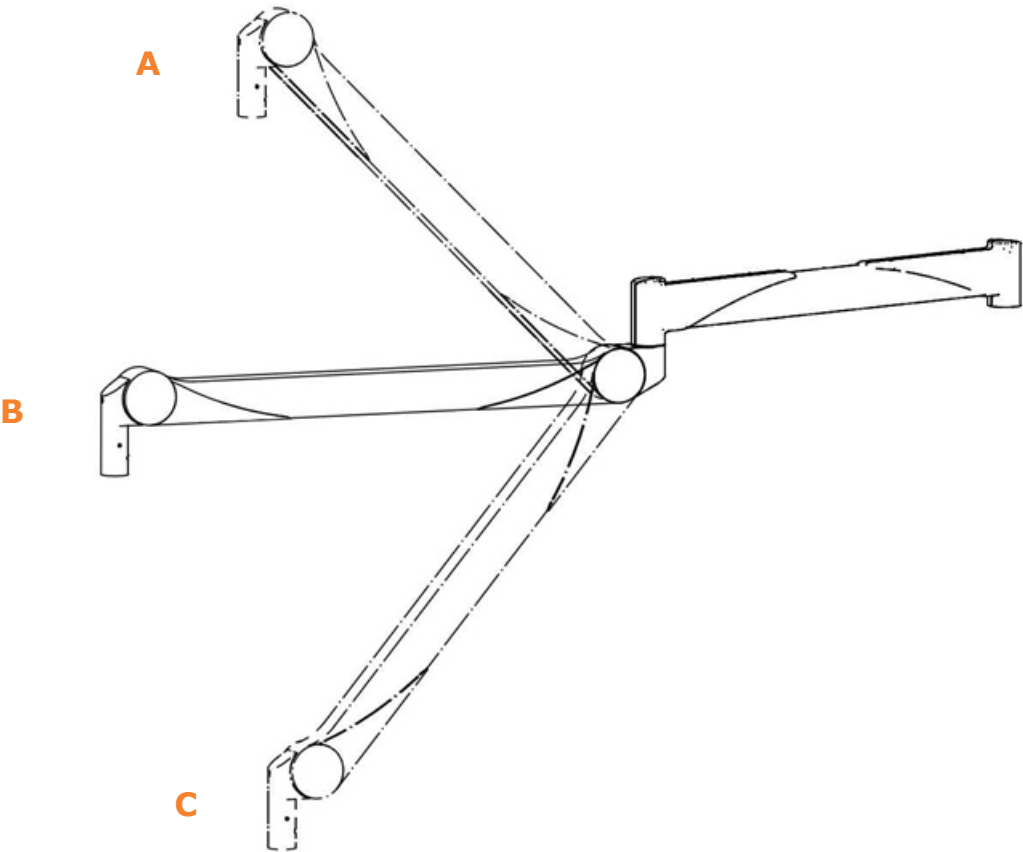
Fig. 26

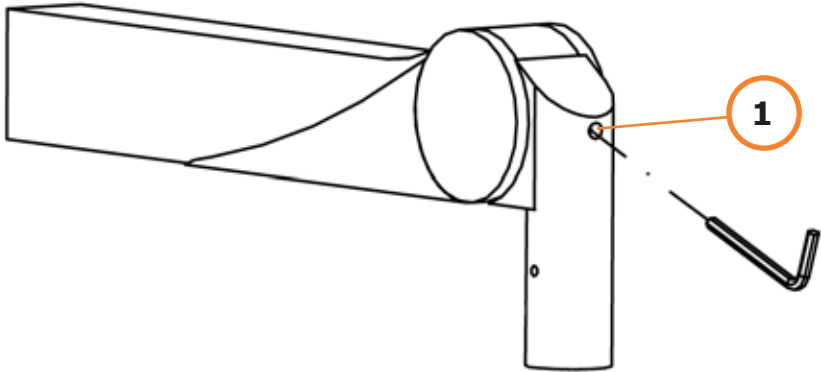
Fig. 27

10.0 Installation and removal of adaptable elements

10.2 Installation of a Surgical Lamp to the Extension/Spring Arm

NOTE: It is strongly recommended that at least two people perform the installation.

WARNING - Electric Shock.

Cut off all electricity prior to performing any works on the system - hazardous situation!

WARNING - Electric Shock.

All MAVIG lamps are 24 V – ALWAYS MAKE SURE TO USE AN APPROPRIATE DELIVERED TRANSFORMER (int./ext.) OR POWER SUPPLY UNIT!

NOTE 01: The M130 has the transformer built into the lamp housing, no additional transformer is necessary!

NOTE 02: All other halogen lights have external transformers, which need to be installed. Specific information can be found in the halogenlamp manuals.

NOTE 03: The LED lights DO NOT have transformers, but external power units, which need to be installed. Specific information is supplied with the LED lamp!

CAUTION - Improper Portegra2 Arm.

Ensure that the arm used is electrical and has an electric socket in the front and end of the arm. Otherwise, no electricity can reach the end device.

WARNING - Under Constant Spring Tension.

Set the upper stop of the spring arm to the horizontal position (Position B/Fig. 26) during the installation process. To correct this, insert an 5 mm allen wrench into the opening (1/Fig. 27) and turn it fully to the right, otherwise the spring arm will uncontrollable swing upwards.

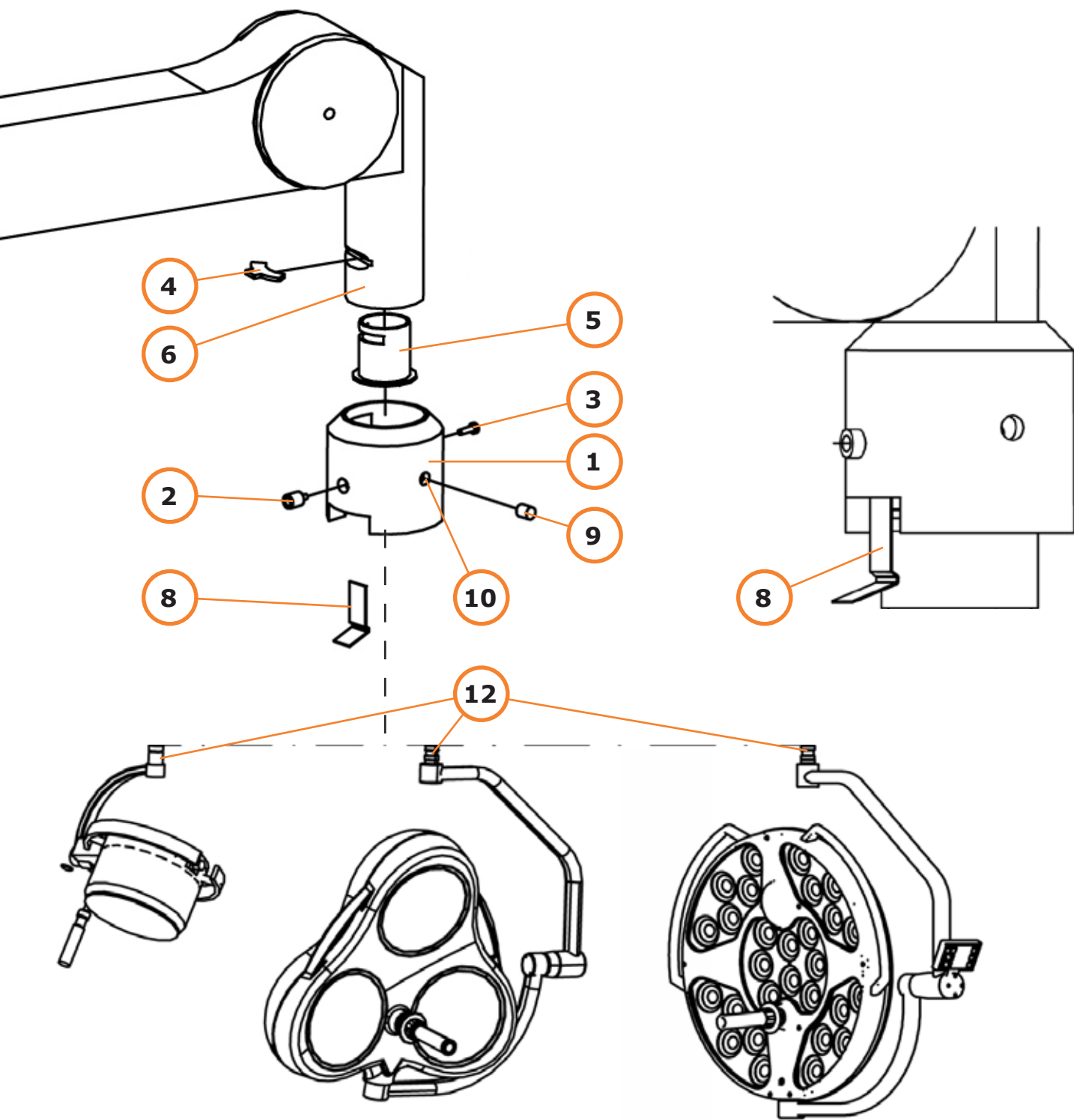
Fig. 28

10.0 Installation and removal of adaptable elements

10.2 Installation of a surgical lamp to the extension/spring arm

10.2.1 Remove the screws (2,3/Fig. 28) from the safety collar (1/Fig. 28). Store them in a safe place.

NOTE: It is best to use a screwdriver with a magnetic tip to loosen the screws.

10.2.2 Push up the safety collar (1/Fig. 28) until it rests on the plastic cover and hold it in this position.

CAUTION - Falling Parts.

Hold the safety collar (1/Fig. 28) to ensure that it will not fall off the spring arm.

10.2.3 Remove the retaining clip (4/Fig. 28) from its slot Fig. 28 and remove the protective cap (5/Fig. 28) from the receptacle (6/Fig. 28).

10.2.4 Insert the mounting pin/connecting part of the adaptable element (12/Fig. 28) into the receptacle (6/Fig. 28) until it rests on its flat surface. Insert the retaining clip (4/Fig. 28) in its slot.

CAUTION - Long End First.

Remove the safety spring (8/Fig. 28) from your shipping carton and insert it, long end first, into the safety collar (1/Fig. 28) as shown in detail (Fig. 28).

NOTE: Please, always use a new MAVIG safety spring. You will find a replacement spring in the shipping carton.

10.2.5 Secure the safety spring (8/Fig. 28) under the retaining clip (4/Fig. 28).

10.2.6 Carefully, slide the safety collar (1/Fig. 28) over the safety spring (8/Fig. 28), until it is flush with the flat surface of the receptacle. While sliding down the safety collar, make sure the safety spring is firmly set.

10.2.7 Insert and tighten the screw (3/Fig. 28).

WARNING - Rotation Difficulty.

The following step is very important for the long-term performance of the system!

10.2.8 Apply Loctite® 270 (included in package) to the set screw (2/Fig. 28) and screw it into the safety collar as far as possible. The lamp holder will be difficult to rotate. Now, back out the screw with a 90° rotation.

10.2.9 Remove the plastic plugs (9/Fig. 28) from your shipping carton and insert them into the drill holes (10/Fig. 28) on the sides of the safety collar (1/Fig. 28).

Portegra2 Spring Arm End Positions

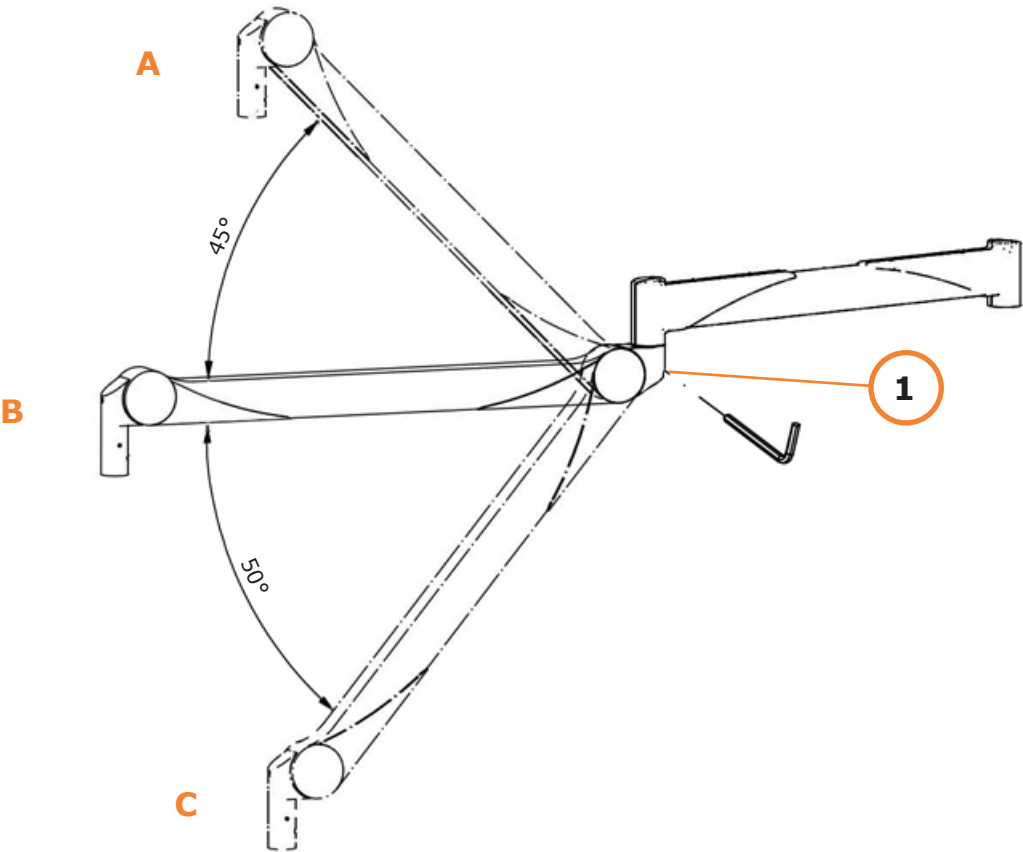
Fig. 29

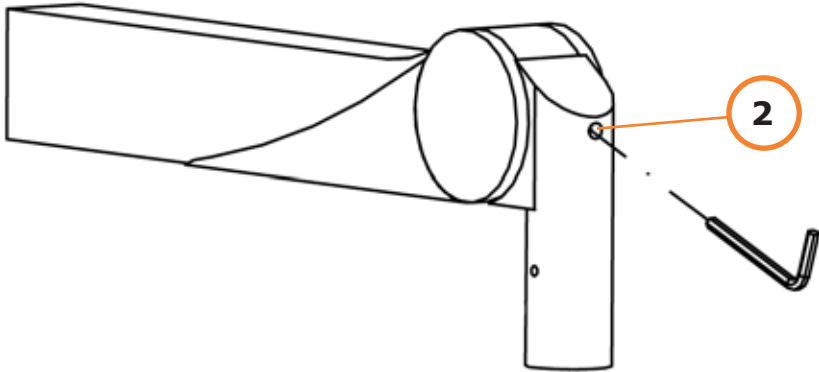
Fig. 30

10.0 Installation and removal of adaptable elements

10.3 Adjusting the weight capacity of the arm

The weight capacity of the spring arm has already been set at the factory for the end device it was purchased with. This max. weight of the spring arm can be found on the product label on top of the extension arm. In rare circumstances or over time the spring arm may need to be slightly adjusted.

- 10.3.1 Using a 5 mm Allen wrench insert it into the weight adjustment slot at the back end of the spring arm (extension arm/spring arm joint, 1/ Fig. 29). Turn the wrench either left (counter-clockwise) or right (clockwise) to adjust the weight.

To increase load capacity, turn the wrench to the left. (Overturning will cause the arm to move upward). To decrease load capacity, turn the wrench to the right. (Overturning will cause the arm to move downward).

- 10.3.2 The spring arm is correctly set when the vertical movement is easy and light, but holds its position after movement.

10.4 Adjusting the vertical movement of the spring arm

- 10.4.1 The spring arm section of the Portegra2 suspension arm can be pulled downwards 50° and moved upwards 45° (Fig. 29). The upper movement of the spring arm can be limited to any point from 0° to 45° by adjusting the arm spring at the upper vertical adjustment slot (2/ Fig. 30).

Using a 5 mm allen wrench, insert it in the slot and turn it to obtain the desired stop point.

- To increase the vertical motion, turn the allen wrench to the left (counter-clockwise)
- To decrease the vertical motion, turn the allen wrench to the right (clockwise)

Portegra2 Spring Arm End Positions

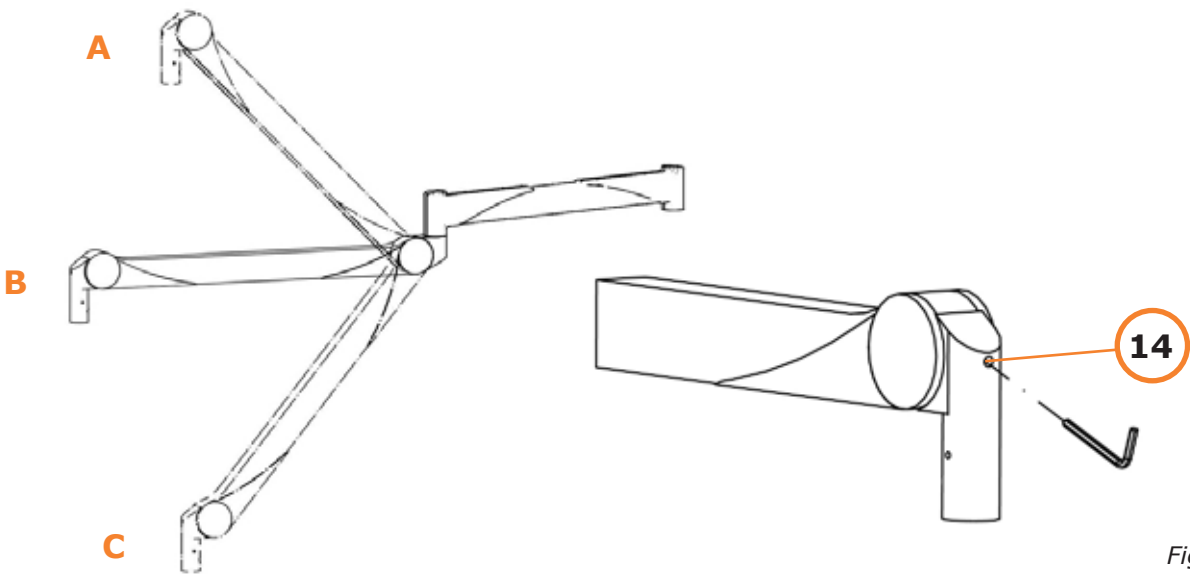
Fig. 31

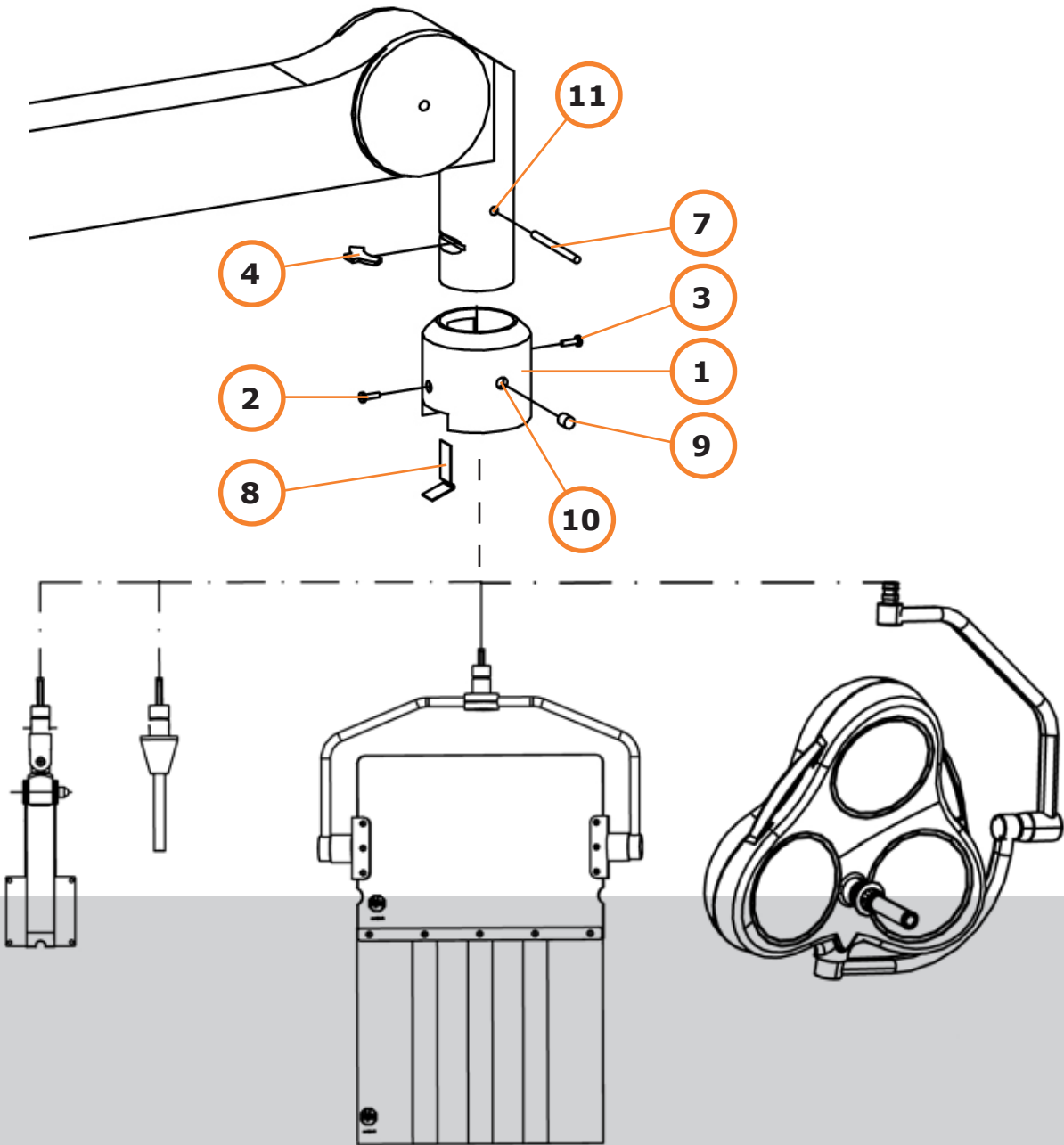
Fig. 32

10.0 Installation and removal of adaptable elements

10.5 Removal of a Device from the End of the Suspension Arm

NOTE: It is strongly recommended that at least two people perform the installation.

WARNING - Electric Shock.

Cut off all electricity prior to performing any works on the system - hazardous situation!

WARNING - Under Constant Spring Tension.

Set the upper stop of the spring arm to the horizontal position (Position B/Fig. 31) during the installation process. To correct this, insert an allen wrench into the opening (14/Fig. 31) and turn it fully to the right, otherwise the spring arm will uncontrollably swing upwards.

NOTE: To adjust the vertical motion, hold the spring arm approximately 10° underneath the horizontal position. This will reduce the pressure on the screw and you can now easily reset it with an allen wrench.

10.5.1 Remove the plastic plugs (9/Fig. 32) from both sides of the safety collar (1/Fig. 32).

10.5.2 Remove the screws (2,3/Fig. 32) from the safety collar and store them in a safe place.

NOTE: It is best to use a screwdriver with a magnetic tip to loosen the screws.

CAUTION - Falling Parts.

Hold the safety collar to ensure that it will not fall off the spring arm while removing an end device.

10.5.3 *For end devices without internal electrical cabling (ex. shields, monitor holders, injector holders):* Push the safety collar upwards (1/Fig. 32), until you will be able to see the straight pin sitting in the drill hole (11/Fig. 32). Remove the straight pin (7/Fig. 32) by pushing it out from one side to the other.

10.5.4 Push up the safety collar (1/Fig. 32) until it rests under the plastic cover and hold it there.

10.5.5 Carefully, pull out the safety spring (8/Fig. 32) from the safety collar (1/Fig. 32).


WARNING - Falling Device.


In the following step, the last thing (retaining clip (4/Fig. 32) inside the spring arm) securing the attached device, is being removed. If you fail to hold the attached device firmly, it will fall down causing damage or injury.

10.5.6 Hold the attached device with a firm grip. Remove the retaining clip (4/Fig. 32) from its slot and disconnect the device. To facilitate the removal, slightly rotate the device if needed.


Portegra2 - 360° Column - Track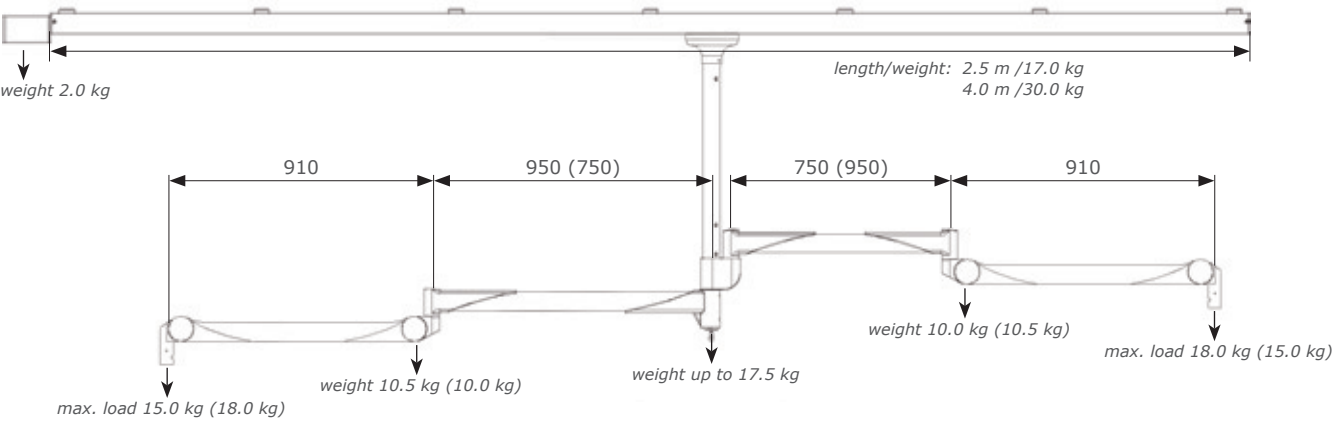


Portegra2 - Twin Column - Track

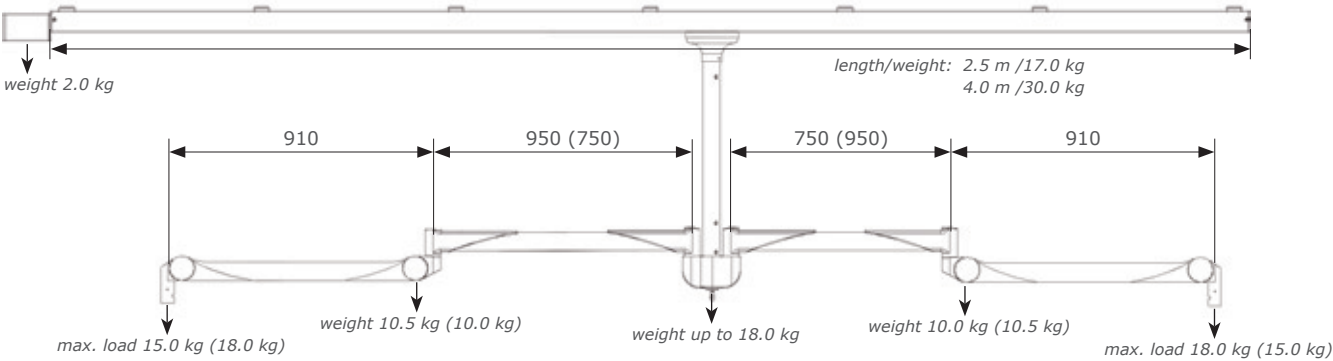Portegra2 - 360° Column - Stationary

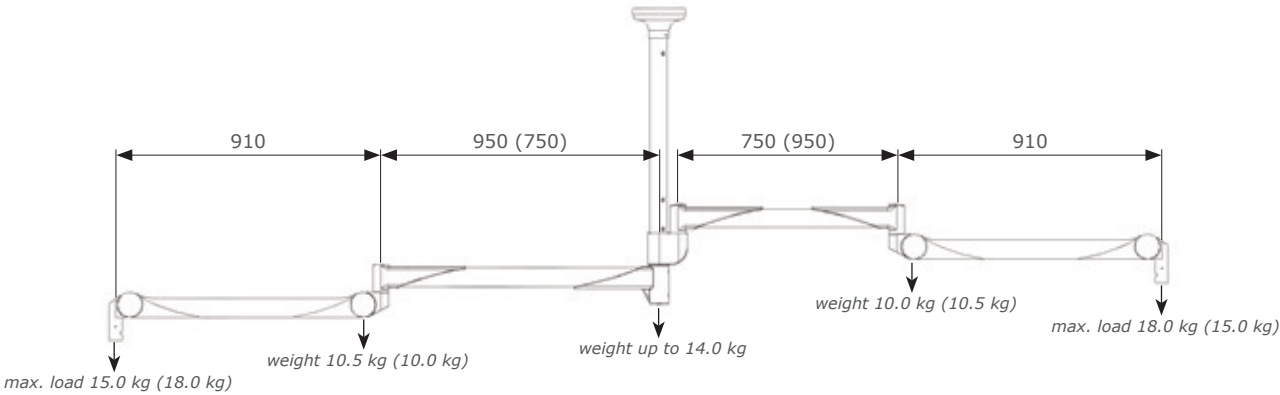Portegra2 - Twin Column - Stationary

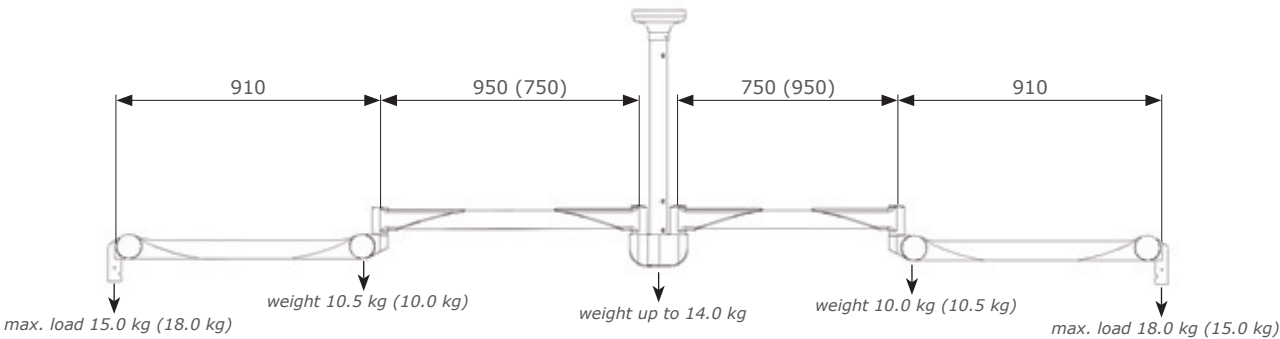11.0 Technical Data

11.1 Portegra2 Component Weights

Component	Description	Length	Weight
Track	2.5 m Long Track	2500 mm / 98.43 in	17.0 kg / 37.48 lbs
	4.0 m Long Track	4000 mm / 157.48 in	30.0 kg / 66.14 lbs
	Cable Spooler		2.0 kg / 4.41 lbs
Ceiling Column	Twin Column with Trolley	460 mm / 18.11 in	15.5 kg / 34.17 lbs
		580 mm / 22.83 in	16.0 kg / 35.27 lbs
		800 mm / 31.50 in	18.0 kg / 39.68 lbs
	360° Column with Trolley	570 mm / 22.40 in	16.0 kg / 35.27 lbs
		800 mm / 31.50 in	17.5 kg / 38.58 lbs
	Twin Column Stationary	460 mm / 18.11 in	10.5 kg / 23.15 lbs
		580 mm / 22.83 in	11.5 kg / 25.35 lbs
		650 mm / 25.59 in	12.0 kg / 26.46 lbs
		850 mm / 33.46 in	13.5 kg / 29.76 lbs
		1000 mm / 39.37 in	14.0 kg / 31.96 lbs
	360° Column Stationary	580 mm / 22.83 in	10.5 kg / 23.15 lbs
		650 mm / 25.59 in	11.5 kg / 25.35 lbs
		850 mm / 33.46 in	13.0 kg / 28.66 lbs
		1000 mm / 39.37 in	14.0 kg / 30.86 lbs
Ceiling/ Wall Mounting	Standard Portegra2 mounting plate Wall mount		22.0 kg / 48.50 lbs
			2.0 kg / 4.41 lbs
Suspension Arm	Extension arm 750 mm + Spring arm 910 mm		10.0 kg / 22.05 lbs
	Extension arm 950 mm + Spring arm 910 mm		10.5 kg / 23.15 lbs

11.2 Suspension Arm Load Capacity

(depending on expansion arm length)

Ceiling Mounted	with Extension arm 750 mm	max.	18.0 kg / 39.68 lbs
	with Extension arm 950 mm	max:	15.0 kg / 33.07 lbs
Wall Mounted	with Extension arm 750 mm	max.	13.0 kg / 28.66 lbs
	with Extension arm 950 mm	max.	10.0 kg / 22.05 lbs

11.3 Possible Spring Arm Load Classes

(Max. weight capacity noted on rating plate on suspension arm)

3.5 - 7.0 kg / 6.0 - 11.0 kg / 7.0 - 12.0 kg / 9.0 - 15.0 kg / 12.0 - 18.0 kg
 7.7 - 15.4 lbs / 13.2 - 24.3 lbs / 15.4 - 26.5 lbs / 19.8 - 33.1 lbs / 26.5 - 39.7 lbs

11.0 Technical Data

11.4 Additional Portegra2 characteristic

Electrical Data	See rating plate
Colour	RAL 9002
Noise, Sound Energy	> 60db(A) (EN ISO 3744) are not exceeded
Operation, Manual Force	< 55 N
Classification	Class 1
IP classification according to IEC60529	IP 20
Protection class in according to EN 60601-1	Class 1

11.5 Standards

Medical Devices Act (Medizinproduktegesetz – MPG)
MDD 93/42/EEC (2007) – Medical Device Directive
IEC 60601-1:2005-12 (Edition 3.0)

11.6 Approved products

The Portegra2 series has been approved for the use of MAVIG OT series radiation protective shields, MAVIG GD40 and GD42 Series monitor holders, and MAVIG GD series device docks/holders, Dr. MACH halogen and LED operation and observation lamps, as well as any other MAVIG products where its compatibility is declared in the devices instructions for use.

Placing a third party device or adaptor onto the Portegra2 system is not allowed without the written consent of MAVIG. Any party putting a third party device into operation with a Portegra2 system or component is responsible for any required risk analysis and or conformity declaration of the system. Such party is also responsible for providing an instructions for use for the operation the end device or adaptor.

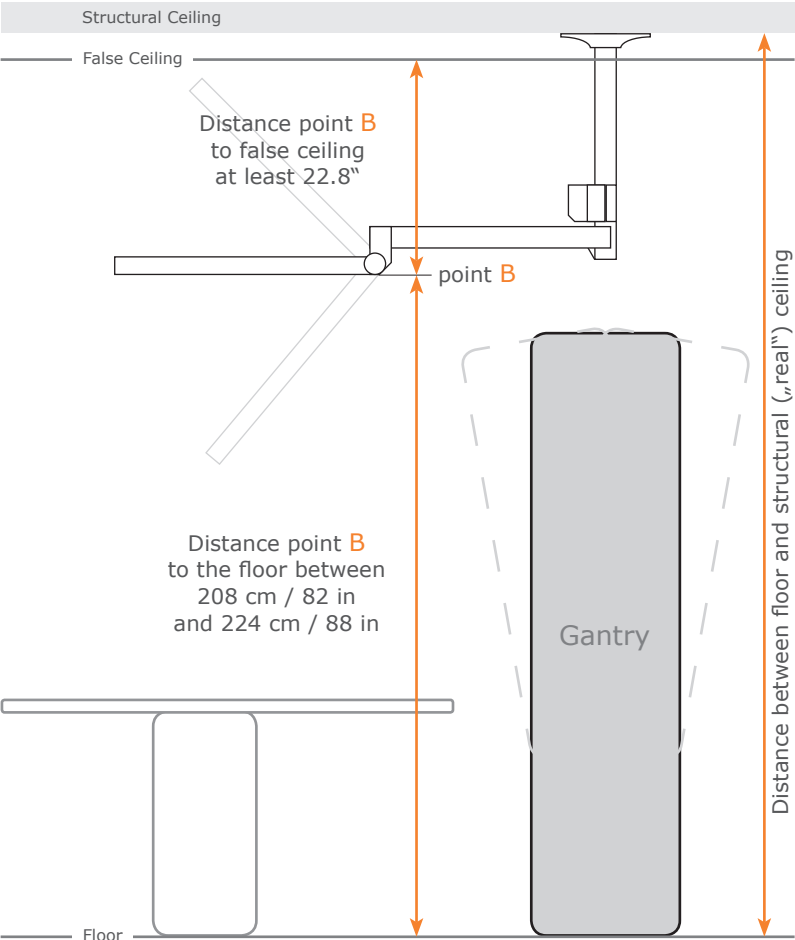
Fig. 31

Chart: Suggested Ceiling Columns

The distance between floor and point B should be approx. 208 cm / 82 in, unless your gantry is higher.

Distance between floor and structural („real”) ceiling	Recommended ceiling column length
220.0 cm - 270.0 cm 86.6 in - 106.3 in	460 mm / stationary 460 mm / track
266.0 cm - 290.0 cm 104.7 in - 114.2 in	580 mm / stationary 580 mm / track
288.0 cm - 310.0 cm 113.4 to 122.0	850 mm / stationary 800 mm / track
300.0 cm - 320.0 cm 118.1 in - 126.0 in	1000 mm / stationary (n.a. with track)

Chart/Fig. 32

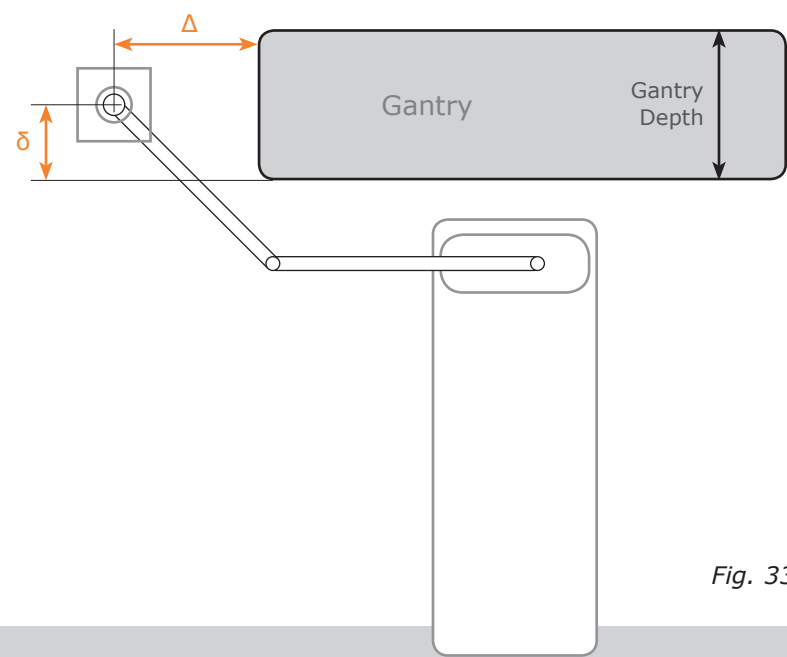
Fig. 33

Chart: Recommended horizontal distance (Δ)

Gantry Depth	Δ – Value
91.4 cm / 36.0 in 94.0 cm / 37.0 in	27.9 cm / 11.0 in 27.9 cm / 11.0 in
96.5 cm / 38.0 in 99.1 cm / 39.0 in	27.9 cm / 11.0 in 30.5 cm / 12.0 in
101.6 cm / 40.0 in 104.1 cm / 41.0 in	30.5 cm / 12.0 in 30.5 cm / 12.0 in

Chart/Fig. 34

12.0 CT Room Planning Considerations/Guide

This chapter provides suggestions on the dimensions, pivot points, and degrees of freedom of each pivot point. This information should be reviewed prior to locating the Portegra2 and selecting the ceiling column.

NOTE: The information provided below for locating the Portegra2 system within the Scanner room and selecting the ceiling column has been provided for guidance.

Consult with the scanner (gantry) manufacturer prior to selecting the ceiling column.

There are three dimensions (I,II,III/ Fig. 31,33) to consider when locating the Portegra2 system:

I) the distance from the ceiling **B** II) the vertical distance **δ** III) horizontal distance **Δ**

12.1 Distance from the ceiling

The optimal distance from the ceiling is for the low point of the Portegra2 (point B, Fig. 31) to be a few cm / inches above the top of the gantry, so that when the gantry is pivoted it does not interfere with the system. A distance between 208 cm / 82 in and 224 cm / 88 in from the floor to point B is typically used to satisfy this requirement.

Consult with the scanner (gantry) manufacturer prior to selecting the ceiling column.

A chart has been developed allowing you to select a ceiling column after taking some measurements of the room. Please see Chart/ Fig. 32 for selection of the recommended ceiling column based on the distance from the floor to the structural ceiling.

12.2 Ceiling column selection

The selections provided in Chart/ Fig. 32 ensure approx. 208 cm / 82 in from the floor to point B on the Portegra2, unless your gantry is higher. If there is a false ceiling, be certain to measure all the way up to the structural („real”) ceiling or UniStrut® that will be used to mount the ceiling plate. Please refer to Fig. 31.

For rooms higher than 320 cm / 126 in, we strongly suggest a ceiling construction between structural ceiling and vertical column.



12.3 Ceiling plate placement (stationary version)

12.3.1 The recommended vertical distance (δ) with respect to the gantry is centered on the gantry depth. Placement is shown in Fig. 33.


12.3.2 The recommended horizontal distance (Δ) with respect to the gantry is to be offset to either side of the gantry by an amount called Δ . This dimension is shown in Fig. 33. The value of Δ is based on the gantry depth, and is calculated to maximize the swing of the Portegra2 arm. Keeping the Portegra2 Δ cm / inches away from the gantry helps to protect both the Portegra2 and gantry, and allows for unencumbered pivot of the gantry. Please see Fig. 33. Chart/ Fig. 34 shows the Δ values for various gantry depths.

12.4 Track placement


The position of the track must be decided locally due to any other ceiling suspended devices or obstacles in the room. Make sure the potential for collisions with other objects has been eliminated.

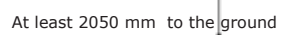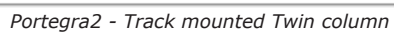Portegra2 - Stationary mounted 360° column

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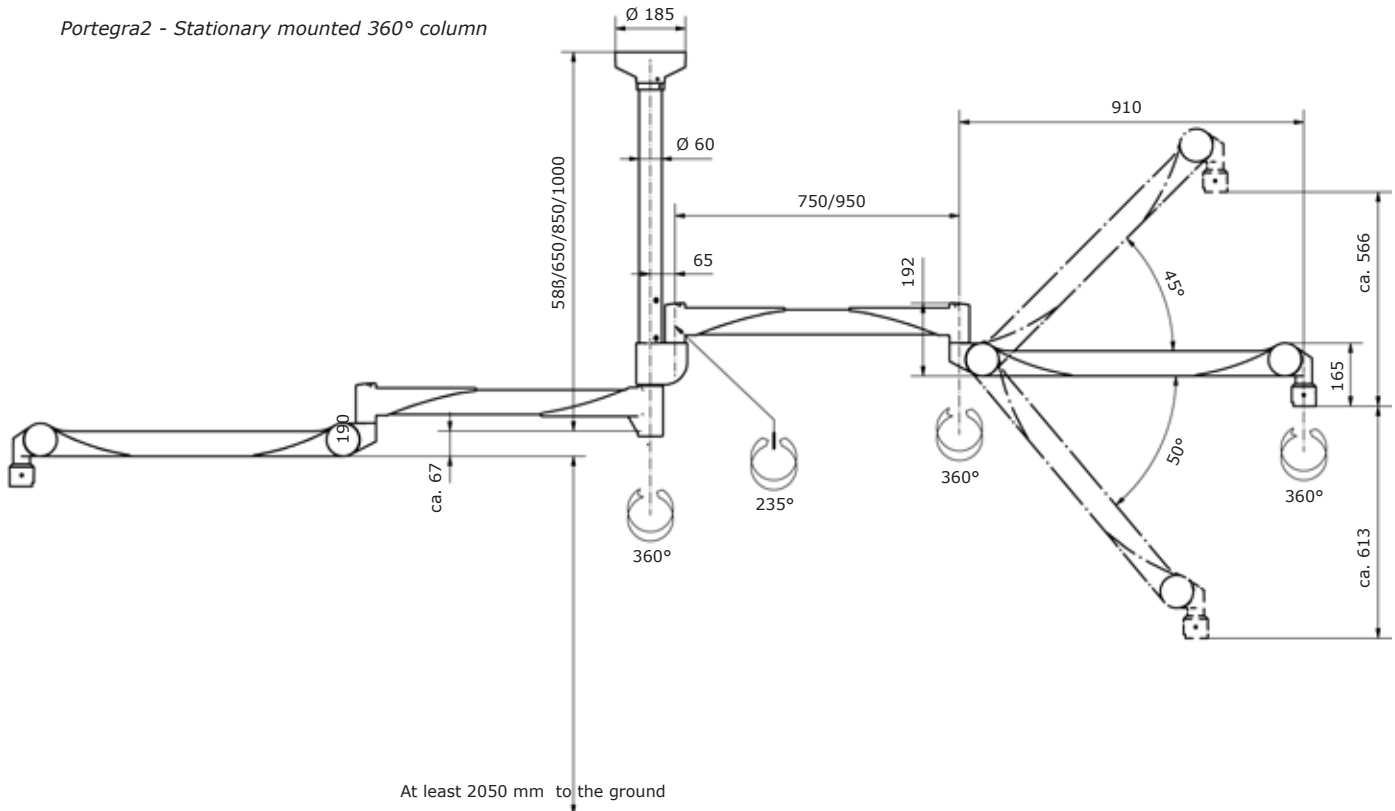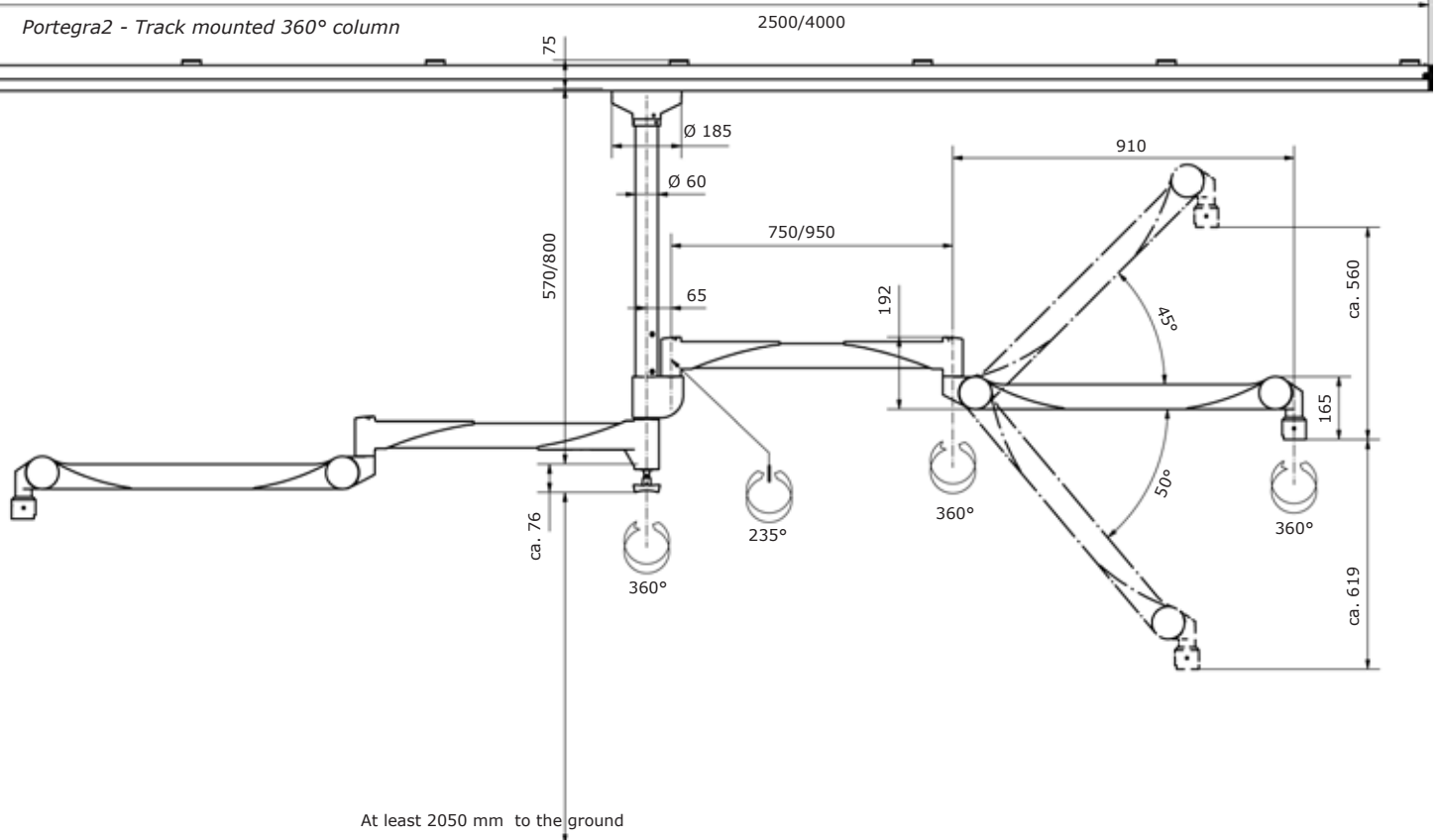The logo for Portegra2, featuring the word "PORTEGRA" in a bold, grey, sans-serif font, followed by a large, stylized orange "2".

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